EN 3	Chait Access Adapter Instructions for Use
CS 3	Přístupový adaptér Chait Návod k použití
DA 4	Chait adgangsadapter Brugsanvisning
DE 5	Chait Zugangsadapter Gebrauchsanweisung
EL 6	Προσαρμογέας πρόσβασης Chait Οδηγίες χρήσης
ES 7	Adaptador de acceso Chait Instrucciones de uso
FR 8	Adaptateur d'accès Chait Mode d'emploi
HU 8	Chait hozzáférési adapter Használati utasítás
IT 9	Adattatore per accesso Chait Istruzioni per l'uso
NL 10	Chait toegangsadapter Gebruiksaanwijzing
NO 11	Chait-tilgangsadapter Bruksanvisning
PL 12	Adapter dostępowy Chait Instrukcja użycia
PT 13	Adaptador de acesso Chait Instruções de utilização
SV 14	Chait åtkomstadapter Bruksanvisning

CHAIT ACCESS ADAPTER

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Chait Access Adapter is a single-patient, reusable connecting tube with a flared fitting on one end and a metal cannula joined to a plastic block at the other end.

INTENDED USE

The Chait Access Adapter is intended to allow connection between the Chait Cecostomy Catheter and the delivery tubing of an irrigation/enema bag system.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

- The Chait Access Adapter should only be used in patients who have a Chait Cecostomy Catheter in place.
- The potential effects of phthalates on pregnant/nursing women or children have not been fully characterized and there may be concern for reproductive and developmental effects.

INSTRUCTIONS FOR USE

1. Prepare the irrigation/enema solution according to healthcare provider recommendations
2. Fill the irrigation/enema delivery system with prepared solution, leaving delivery tubing clamped.
3. Connect the flared fitting of the Chait Access Adapter to the irrigation/enema delivery system tubing.
4. Open the clamp on the delivery system tubing and fill the tubing and adapter with prepared solution.
5. Close the clamp on the delivery system tubing.
6. Open the Chait trapdoor mechanism.
7. Connect the Chait Access Adapter to the open Chait trapdoor.
8. Insert the metal cannula of the adapter into the lumen of the catheter and align the hole in the plastic block with the pin in the trapdoor cover.
9. Advance the adapter until the pin and cannula are fully seated.
10. Open the clamp and allow irrigation/enema solution to flow through the Chait catheter into the bowel.
11. Once irrigation/enema is complete, disconnect the irrigation/enema set and Chait Access Adapter from the Chait catheter.
12. Close the Chait catheter trapdoor fitting.
13. Clean the Chait Access Adapter and the irrigation/enema set with a solution of warm soapy water, rinse until no soap remains, and allow to dry before storing.
14. Store the Chait Access Adapter at room temperature in a clean, dry location.

HOW SUPPLIED

Supplied in individual peel-open packages sterilized by ethylene oxide gas. Intended for single patient use. Upon removal from package, inspect the product to ensure no damage has occurred. Product may be cleaned and re-used for a single patient.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK 2019
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd