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T _ C C I S _ R E V O

COOK-COPE LOOP CATHETER INTRODUCTION SET

Used to introduce Cook-Cope loop and other catheters for percutaneous nephrostomy. Intended for one-time use.

SET CONTAINS

- 21 gage stainless steel thinwall needle 15 cm long
- .018 inch (0.46 mm) diameter stainless steel Cope mandril wire guide 60 cm long with flexible coude tip
- 6.3 French radiopaque polyethylene introducing catheter 22 cm long
- 20 gage stainless steel stiffening cannula 22 cm long
- .038 inch (0.97 mm) diameter TFE coated stainless steel Safe-T-J® wire guide 100 cm long with 3 mm Safe-T-J® tip

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

There are no known contraindications.

SUGGESTED INSTRUCTIONS FOR USING COOK-COPE LOOP CATHETER INTRODUCTION SET

1. Introduce the 21 gage thinwall needle.
2. Pass the flexible tip of the .018 inch (0.46 mm) diameter wire guide through the needle.
3. Leaving the wire guide in place, withdraw the needle.
4. With a twisting motion, advance the 6.3 French catheter with its 20 gage cannula stylet in place, over the wire guide.
5. Withdraw the wire guide and 20 gage cannula stylet.
6. Pass the straightened flexible tip of the .038 inch (0.97 mm) diameter Safe-T-J® wire guide through the catheter (the wire guide will exit through the single sideport of the catheter).
7. Withdraw the 6.3 French catheter, leaving the .038 inch (0.97 mm) diameter wire guide in place.

REFERENCES

C. Cope, R. M. Zeit: "Pseudoaneurysms After Nephrostomy," American Journal of Roentgenology, 139 (1982), 255-261.

C. Cope: "Replacement of Obstructed Loop and Pigtail Nephrostomy and Biliary Drains," American Journal of Roentgenology, 139 (1982), 1022-1023.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.