Disposable Hysteroscopic Polyp Snare

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T _ D H P S _ R E V O

DISPOSABLE HYSTEROSCOPIC POLYP SNARE

Used to cut and coagulate polyps and fibroids for removal from the uterus. Can be used through a rigid or flexible hysteroscope, with or without electrocautery. Intended for one-time use.

DESCRIPTION

Sheath 50 cm long with stainless steel duckbill snare and attached 3-ring handle.

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is damaged.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

This device should not be used on a patient with an active vaginal or intrauterine infection, a sexually transmitted disease, a recent uterine perforation, a recent cesarean section, who is currently pregnant, or if the patient currently has, or suspects she may have, an intrauterine device within her uterus.

CAUTION: The Hysteroscopic Polyp Snare is intended for use by physicians trained and experienced in transcervical and hysteroscopic procedures, including but not limited to hysteroscopic polypectomy.

IMPORTANT: Excessive force could damage device.

SUGGESTED INSTRUCTIONS FOR USING HYSTEROSCOPIC POLYP SNARE

1. Hook active cord into handle of snare.

WARNING: Always make sure that a good return path to the electrosurgical unit is maintained.

WARNING: Do not use a device that has been cut, burned or damaged. Damaged insulation may cause unsafe currents in either patient or operator.

WARNING: Any electrosurgical accessory constitutes a potential electrical hazard to the patient and operator. Possible adverse effects include: fulguration, burns, stimulation, and cardiac arrhythmia.

NOTE: Electrosurgical devices have a maximum voltage rating of 2000 Vp-p in pure cut mode and 4000 Vp-p in coagulation mode.

2. Retract the snare into the sheath.

3. Insert the snare assembly through the accessory channel of the hysteroscope until the tip is visualized in the uterus. **NOTE:** A nonionic iso-osmolar irrigant must be used.

4. Open snare and place around tumor.

5. Draw snare down around the base of tumor and retract snare into sheath. This may be accomplished with or without electrocautery.

NOTE: Gentle traction towards the hysteroscope reduces the risk of uterine trauma and tumor fragmentation while providing a clean base to obtain uterine biopsies.

6. After the tumor has been cut, remove the snare and then remove the tumor with forceps or graspers.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.