EN
2

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Bruksanvisning

T _ D U A N 2 _ R E V 0

PIGTAIL DRAINAGE CATHETER NEEDLE SETS

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

The Pigtail Drainage Catheter Needle Set contains a polyurethane pigtail catheter with pigtail straightener (if applicable) and a needle set consisting of a cannula, trocar point stylet, and loading obturator, all made of stainless steel.

INTENDED USE

The Pigtail Drainage Catheter Needle Set is intended for percutaneous external drainage in multiple drainage applications (e.g., nephrostomy, abscess drainage and other abdominal cavity drainage procedures).

CONTRAINDICATIONS

None known

WARNINGS

If the catheter becomes malpositioned or if drainage ceases, the catheter should be promptly exchanged or removed.

PRECAUTIONS

- This product is intended for use by healthcare practitioners trained and experienced in diagnostic and interventional techniques. Standard techniques for direct placement of percutaneous drainage catheters should be employed.
- Fluoroscopy or ultrasound is recommended during needle placement.
- A TFE-coated wire guide is recommended for catheter introduction.
- When using the catheter needle set, make sure not to damage the catheter during introduction of the needle.
- Do not reintroduce the trocar stylet after catheter placement.
- Catheters should be irrigated periodically to ensure function.
- Patients with indwelling catheters should be checked on an outpatient basis to ensure continuous function of the catheter.

MRI SAFETY INFORMATION

The Pigtail Drainage Catheter is **MR Safe** according to ASTM F2503. The accessory components have not been tested for MRI compatibility.

INSTRUCTIONS FOR USE

1. To straighten the pigtail of the catheter, slide the Peel-Away® straightener forward, if applicable.
2. Insert the cannula with the loading obturator and lock it to the catheter.
3. Peel off the straightener, if applicable.
4. Remove the loading obturator, insert the stylet, and lock it to the cannula.
5. Under fluoroscopic control or by ultrasound-guided puncture, introduce the assembled drainage set into the fluid collection. Dilation is not necessary.
6. Remove the trocar stylet. By simple suction or contrast injection, check the position of the catheter tip.
7. Advance the catheter from the cannula until the pigtail is formed. Remove the cannula. Fix the catheter to the skin and connect the catheter to a drainage bag.
8. To remove or exchange the catheter, introduce a stiff wire guide such as the Amplatz Ultra Stiff or the Lunderquist Extra Stiff type to the distal end of the pigtail under fluoroscopic control. Holding the wire guide in a stable position, withdraw the catheter.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd