Dretler Ureteroscopy Stent Set

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T _ D U S S _ R E V O

DRETLETER URETEROSCOPY STENT SET

Used for post-ureteroscopy stenting of the ureter. The stent is positioned through the ureteroscope prior to removal of the scope. Intended for one-time use.

SET CONTAINS *(set components may vary)*

- Single pigtail ureteral stent
- Appropriate diameter stainless steel wire guide
- Tuohy-Borst adapter

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CONTRAINDICATIONS

There are no known contraindications.

SUGGESTED INSTRUCTIONS FOR USING DRETLETER URETEROSCOPY STENT SET

Filiform Placement Method

1. Pass the wire guide, flexible end first, through the back end of the ureteral stent through the coil until it is straight, leaving 3 to 4 cm of the wire guide extending beyond the tip.
2. Place the stent/wire guide assembly through the ureteroscope and position at the desired location in the renal pelvis.
3. Remove wire guide. The stent pigtail will form spontaneously.
4. Check for full coil in the renal pelvis.
5. Slide Tuohy-Borst adapter onto the externalized portion of the stent and tighten.
6. If necessary, use a connecting tube to connect the stent to a drainage bag.

NOTE: The correct position of the stent may be confirmed by radiographic means.

NOTE: The stent may be placed over existing working wire guide and pushed into position in the renal pelvis. Repeat steps 3 through 6.

CAUTION: The maximum indwelling time is six (6) months.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.