**EN**  
3

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**Endoskopskydd**  
Bruksanvisning

## ENDOSCOPIC CAP

**CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner). Please read all instructions before using this device.**

### DEVICE DESCRIPTION

The Endoscopic Cap is an endoscopic adapter designed to facilitate passage of instruments ranging in size up to 12 French in diameter during a procedure. The adapter can be connected to any standard female Luer lock fitting or bulb-type endoscopic fitting.

### INTENDED USE

This adapter is intended to resist the backflow of fluid around an instrument inserted through the working channel of a ureteroscope, cystoscope, or other endoscope.

### CONTRAINDICATIONS

None known

### WARNINGS

None known

### PRECAUTIONS

This device is intended for use by physicians trained and experienced in urological procedures. Standard techniques for placement of wire guides and devices should be employed.

### INSTRUCTIONS FOR USE

1. Introduce the adapter onto the working channel of an endoscope via a standard female Luer lock fitting or bulb-type fitting.
2. Introduce catheter or other desired instrument through the port of the adapter as necessary to complete the intended procedure.
3. After removal of the inserted device and completion of the procedure, carefully remove the device from the endoscope.

### HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

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Bloomington, IN 47404 U.S.A.