Cook Flexible Biopsy Forceps	EN
Instructions for Use	3
Cook fleksibel biopsitang	DA
Brugsanvisning	5
Flexible Biopsiezange von Cook	DE
Gebrauchsanweisung	7
Εύκαμπτη λαβίδα βιοψίας Cook	EL
Οδηγίες χρήσης	9
Pinzas flexibles de biopsia Cook	ES
Instrucciones de uso	11
Pince souple à biopsie Cook	FR
Mode d'emploi	13
Pinza flessibile per biopsia Cook	IT
Istruzioni per l'uso	15
Cookフレキシブル生検用鉗子	JA
使用方法	17
Cook flexibele biopsietang	NL
Gebruiksaanwijzing	19
Pinça de biopsia flexível Cook	PT
Instruções de utilização	21
Cook flexibel biopsitång	SV
Bruksanvisning	23
Cook 柔性心肌活检钳	ZH
使用说明	CN 25

COOK FLEXIBLE BIOPSY FORCEPS

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

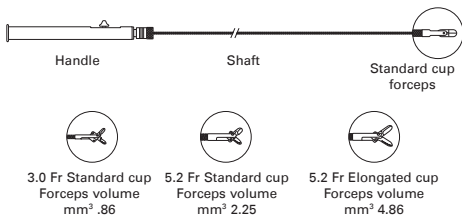
Fig. 1

The Cook Flexible Biopsy Forceps (order number prefix **FBF-**) consists of a handle, a TFE-sheathed, braided stainless steel shaft and a standard cup for tissue sampling. The order number prefix **FBFE-** indicates an elongated cup. (**Fig. 1**)

INTENDED USE

The Cook Flexible Biopsy Forceps is intended for myocardial tissue sampling for biopsy.

The product is intended for use by physicians trained and experienced in diagnostic and interventional techniques. Standard techniques for placement of vascular access sheaths, angiographic catheters and wire guides should be employed.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

- Manipulation of the product requires fluoroscopic control.
- Visually inspect the product before use to make sure the shaft is free of kinks or severe angulation.
- The recommended introducer sheath should be one French size larger than the forceps size.
- Verify the distal tip position of the sheath prior to each biopsy performed if more than one sampling is required.

INSTRUCTIONS FOR USE

1. Under fluoroscopy, advance the forceps through an appropriate sheath introducer or catheter to the biopsy site.
2. When the distal cup of the forceps exits the distal end of the sheath or catheter, move the thumb button on the handle forward to open the biopsy cup.
3. Advance the opened cup to the area where the specimen for biopsy is to be obtained.
4. Retract the thumb button to close the jaws of the cup and secure the tissue specimen.
5. Maintain the thumb button in the retracted position to ensure that the cup remains closed and keeps the specimen intact.
6. Continuously flush heparinized saline through the sheath or catheter while slowly retracting the closed biopsy forceps.

NOTE: If more than one biopsy specimen is required, rinse the forceps with heparinized saline before and after each tissue extraction.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.