Fiber Cleaving Tool

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Glasfaser-Schneidwerkzeug

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FIBER CLEAVING TOOL

SUGGESTED CLEAVING INSTRUCTIONS

1. Place Fiber Cleaver on a flat solid surface.
2. Remove approximately 10 mm of buffer material from the fiber tip using a fiber stripper of appropriate size.
3. Raise and expose the diamond cleaving wheel on the Fiber Cleaver.
4. Wearing rubber gloves, hold the fiber tip and carefully score the fiber core approximately 6mm from the distal dip, perpendicular to the diamond wheel.
5. Gently pull the distal tip straight away from the fiber. Be careful not to bend the fiber tip at the point of separation as this can result in a less even break.
6. Properly dispose of separated quartz tip.
7. Inspect cleaved tip for quality of cleave.
8. If a better cleave is required, repeat steps 2 through 7.

Note: Good cleaves are more easily achieved with small core fibers (i.e. 273 and 365 micron) compared with larger core fibers (550 and 940 micron). Practice with sample fibers is highly recommended, especially with the larger core fibers.

Note: Stripping and cleaving of fibers should be performed in a controlled laboratory environment away from the clinical setting and operating theater.

Note: Basic tips on handling fiberoptic devices:

- Always wear safety eyewear when reprocessing fiberoptic assemblies.
- Never subject fiberoptics to sharp bends in handling, use, storage or reprocessing.
- Always keep connector end dry and free from contaminants.
- It is recommended that only facilities that are adequately designed, equipped, monitored and staffed by trained personnel should undertake surgical fiber reprocessing at hospital site.
- Discard any fiberoptic assembly that is cracked, broken or does not meet minimum transmission standards.
- When in doubt, consult qualified hospital personnel or contact manufacturer at 812-829-4891.

Note: In addition to above recommended fiber handling protocol, all other applicable hospital protocols required to insure patient and hospital personnel safety should be followed.

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© COOK 2013
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.