COOK

MEDICAL

EN
3

Flushing Catheter

Instructions for Use

CS
3

Proplachovací katetr

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DA
4

Skyllekatheter

Brugsanvisning

DE
5

Spülkatheter

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EL
5

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T _ F C _ R E V 2

FLUSHING CATHETER

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Flushing Catheter is a closed-end catheter with a directional side port. The ink mark on the proximal end is positioned in line with the port, and a reference band is positioned 7 cm from the distal tip of the catheter.

INTENDED USE / INDICATION FOR USE

The Flushing Catheter is intended for instillation of sterile medium into the cervical canal prior to embryo transfer.

CONTRAINDICATIONS

Do not use this device in the presence of:

- Active pelvic infection
- Sexually transmitted disease
- Pregnancy
- Confirmed or suspected intrauterine device
- Uterine perforation

WARNINGS

None known

PRECAUTIONS

None known

INSTRUCTIONS FOR USE

1. Place the patient in a lithotomy or knee/chest position.
2. Insert a vaginal speculum to expose the cervix.
3. Use a swab to clean excess cervical mucus.
4. Attach a 10 mL syringe filled with transfer medium to the catheter.
5. Insert the catheter and advance its tip to the internal cervical ostium.
6. Inject approximately 2 to 3 mL while slowly rotating and withdrawing the catheter from the cervix completely.
7. Wait 20 to 30 seconds for the fluid and mucus to exit the cervix. If mucus is still noted within the contrast medium, repeat the previous procedure a second time.
8. Remove the catheter from the patient and dispose of it once the procedure is completed.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.