Fiber Inspection Microscope

Mikroskop na optické kabley

Fiberinspektionsmikroskop

Glasfaser-Inspektionsmikroskop

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LIGHTED CONNECTOR INSPECTION MICROSCOPE (See figure A)

CARE AND MAINTENANCE

- Keep the microscope dry. If it gets wet, wipe it dry immediately. Liquids can contain minerals that corrode the electronic circuits.
- Use only fresh batteries of the recommended size and type. Always remove old or weak batteries. They can leak chemicals that can damage electronic circuits.
- Use and store the microscope only in normal temperature environments. Extreme temperature can shorten the life of internal components and distort or melt plastic parts.
- Occasionally wipe the microscope with a damp cloth to keep it looking new. Do not use harsh chemicals, cleaning solvents, or strong detergents to clean the microscope.

BATTERY REPLACEMENT (See figure B)

- Remove the battery compartment cover by sliding it in the direction of the arrow.
- Insert two AA batteries as indicated by the polarity symbols (+ and -) marked in the battery compartment.
- Replace the battery compartment cover.

BULB REPLACEMENT (See figure C)

- Remove the transparent cover by sliding it sideways and pulling it out.
- Unscrew and remove the old bulb.
- Screw in the new bulb.
- Replace the transparent cover by inserting its four tabs into their respective grooves and sliding the cover into place.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.