**EN**  
4**Flexor® Parallel™ Rapid Release™ Ureteral Access Sheath**

Instructions for Use

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
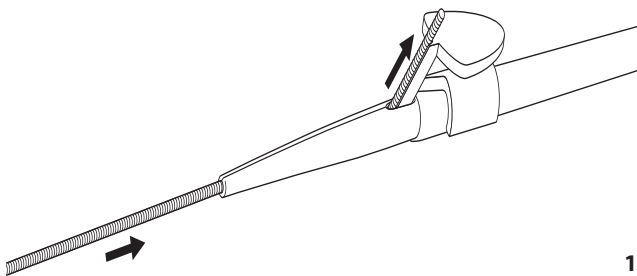
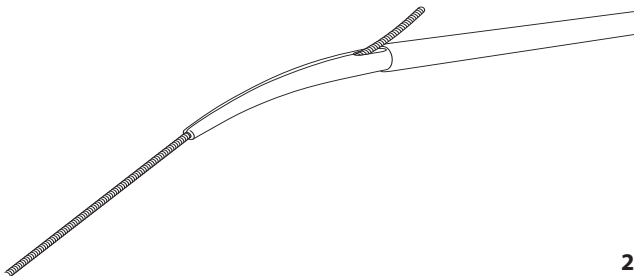
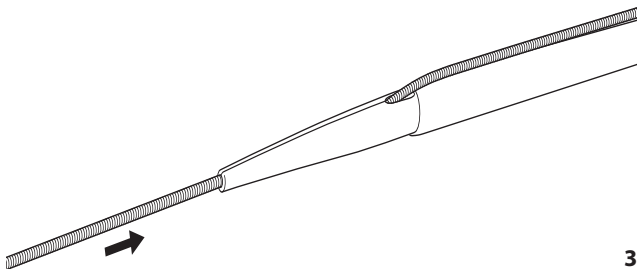
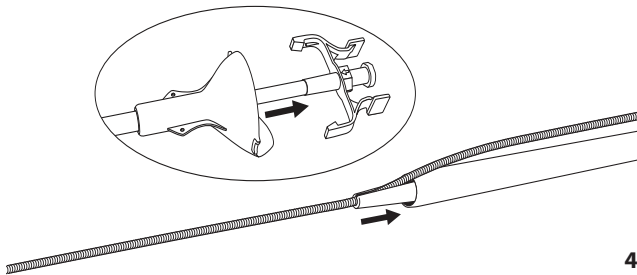
4

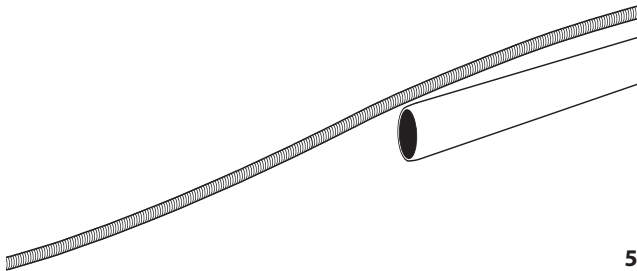5

## FLEXOR® PARALLEL™ RAPID RELEASE™ URETERAL ACCESS SHEATH

**CAUTION:** U. S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

### DEVICE DESCRIPTION

- Ureteral access sheath and tapered dilator with locking hub (AQ® hydrophilic coating is applied to the dilator/sheath assembly to ease placement.)
- Wire guide backloading clip

**NOTE:** Set components may vary.

### INTENDED USE

This device is used to establish a conduit during endoscopic urological procedures, facilitating the passage of endoscopes and other instruments into the urinary tract.

### CONTRAINDICATIONS

None known

### WARNINGS

None known

### PRECAUTIONS

- Never use undue force for placement of this device.
- Never use undue force while releasing the wire in the rapid release technique.
- To reduce the likelihood of trauma, use the rapid release technique once per device.
- During placement, the likelihood of the device slipping off the wire guide increases if a soft wire guide is used. A stiff wire guide is recommended for best performance.

### INSTRUCTIONS FOR USE

**NOTE:** Prior to placement, activate the hydrophilic coating by removing the dilator from the Flexor sheath and immersing all components in sterile water or isotonic saline. This will allow the hydrophilic surface to absorb water and become lubricious, easing placement under standard conditions.

#### Traditional Placement Technique

1. Place a .038 inch (0.97 mm) diameter wire guide the desired length into the ureter to establish a working tract.
2. Ensure the dilator is securely locked onto the instrument adapter, allowing the dilator/sheath assembly to be placed as a single unit with one hand.
3. Grasp the sheath just below the instrument adapter and advance the distal tip of the dilator/sheath assembly over the wire guide and into the ureter.
4. Confirm the dilator/sheath assembly is properly placed via fluoroscopy.
5. While holding the Flexor sheath in position, unlock the fitting and remove the dilator and wire.

**NOTE:** The arrow on the dilator clip indicates the orientation in which the skive is facing. If negotiating tortuous or curved anatomy, rotate the secured dilator and sheath so that the arrow on the clip is facing toward the apex of the curve (away from the curve).

6. Introduce the desired instrument as needed.
7. Suture may be utilized to secure the adapter externally. Suture holes are located on the instrument adapter.

#### Rapid Release Technique

1. Place a .038 inch (0.97 mm) diameter wire guide the desired length into the ureter to establish a working tract. A stiff wire guide is recommended for best results.

2. Ensure the dilator is securely locked onto the instrument adapter, allowing the dilator/sheath assembly to be placed as a single unit with one hand.
3. A wire guide backloading clip is included to aid backloading of soft wires. For soft wires, place the angled tooth of the wire guide backloading clip into the skive and attach the wire guide backloading clip to the sheath. Place thumb on wire guide backloading clip for stability. For stiffer-ended wires, the wire guide backloading clip is unlikely to be necessary.
4. Backload the wire guide through the dilator tip as shown in **Fig. 1** (for soft wires) or **Fig. 2** (for stiff wires). Ensure the wire guide exits out of the skive and runs parallel to the sheath. Once the wire guide is successfully backloaded, remove the wire guide backloading clip (if used) as shown in **Fig. 3**.
5. Grasp the sheath just below the instrument adapter and advance the distal tip of the dilator/sheath assembly over the wire guide and into the ureter.  
**NOTE: The arrow on the dilator clip indicates the orientation in which the skive is facing. If negotiating tortuous or curved anatomy, rotate the secured dilator and sheath so that the arrow on the clip is facing toward the apex of the curve (away from the curve).**
6. Confirm the dilator/sheath assembly is properly placed via fluoroscopy.
7. While holding the Flexor sheath in position, unlock the fitting and remove the dilator in one slow, continuous pull. (**Fig. 4**) Removal of the dilator will release the wire guide via the slit on the distal tip. The wire guide will remain outside the working channel to maintain access. (**Fig. 5**)
8. Introduce the desired instrument as needed.
9. Suture may be utilized to secure the adapter externally. Suture holes are located on the instrument adapter.

## HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

## REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COPYRIGHT COOK 2013
Bloomington, Indiana 47404, U.S.A.
**COOK IRELAND LTD.**