EN  
3

## Fiber Stripping Tool

Instructions for Use

CS  
4

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T \_ F S T \_ R E V 3

## FIBER STRIPPING TOOL

**CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).**

### INTENDED USE

The Fiber Stripping Tool is a non-patient-contact device used to strip the distal end of laser fibers. This device can be used in a sterile field and is capable of being resterilized.

### CONTRAINDICATIONS

None known

### WARNINGS

None known

### PRECAUTIONS

- This product is intended for use by clinicians trained and experienced in laser fiber stripping techniques.
- Always wear safety eyewear when reprocessing fiberoptic assemblies.

### PRODUCT RECOMMENDATIONS

It is recommended that the laser fiber being stripped match the size of the fiber stripping tool being used.

### INSTRUCTIONS FOR USE

1. Prior to stripping, remove all particles and residue from the fiber and clean it per the manufacturer's recommended procedure.
2. Feed the fiber through the front hole to the desired amount of material to be removed.
3. Applying a light amount of pressure, carefully squeeze the grips together.
4. Pull the stripping tool across the fiber using a constant force.
5. Divide longer strip areas up into several segments and repeat this process.

### CLEANING AND STERILIZATION INSTRUCTIONS

1. Place the stripper in a suitable basket inside the cleaning/disinfection machine.
2. Place the loaded basket in the cleaning/disinfection machine so that the water jet directly strikes the stripper.
3. Start the cleaning cycle, including thermal disinfection. Thermal disinfection should be carried out keeping in mind the A0 value and the federal regulation DIN EN ISO 15883-1. The following agents should be used:
  - a. Slightly alkaline cleaning agent
  - b. Neutralizing agent
4. Following the cleaning cycle, remove the stripper from the cleaning/disinfection machine and dry it.
5. Perform a visual inspection for cleanliness and any possible damage.
6. Steam sterilize in a fractional prevacuum autoclave at 134°C for 5 minutes in Europe and at 132°C for three minutes in all other countries.

### HOW SUPPLIED

Supplied non-sterile. Reusable. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

### REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK 2014
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.