COOK
MEDICAL

CE
0123

EN 3	Intrauterine Insemination Catheter Instructions for Use
CS 4	Katetr pro intrauterinní inseminaci Návod k použití
DA 5	Intrauterint inseminationskateter Brugsanvisning
DE 6	Intrauteriner Inseminationskatheter Gebrauchsanweisung
EL 8	Καθετήρας ενδομήτριας σπερματέγχυσης Οδηγίες χρήσης
ES 9	Catéter de inseminación intrauterina Instrucciones de uso
FR 10	Cathéter d'insémination intra-utérine Mode d'emploi
HU 12	Méhen belüli megtermékenyítéshez szolgáló katéter Használati utasítás
IT 13	Catetere per inseminazione intrauterina Istruzioni per l'uso
NL 14	Intra-uteriene inseminatiekatheter Gebruiksaanwijzing
NO 16	Kateter for intrauterin inseminasjon Bruksanvisning
PL 17	Cewnik do inseminacji wewnętrznej Instrukcja użycia
PT 18	Cateter de Inseminação Intrauterina Instruções de utilização
SV 20	Intrauterin inseminationskateter Bruksanvisning

T - I I C - R E V 2

INTRAUTERINE INSEMINATION CATHETER

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

Human Sperm Survival Assay (HSSA) \geq 70% motility after 24 hours, USP Endotoxin Assay (LAL) \leq 20 EU per device, and Sterility Assurance Level [SAL] 10^{-6} .

DEVICE DESCRIPTION

INSEMI-CATH® INSEMINATION CATHETER, SHEPARD INTRAUTERINE INSEMINATION SET, OR SOFT-PASS™ COAXIAL INSEMINATION CATHETER

Intrauterine insemination catheters are comprised of a polymer material. Some catheters have an internal cannula to add rigidity and an adjustable positioner or reference markers to assist in placement. Some catheter sets include a guiding catheter.

INTENDED USE / INDICATION FOR USE

Intrauterine insemination catheters are used for the introduction of washed spermatozoa into the uterine cavity.

CONTRAINDICATIONS

This device should not be used in the presence of:

- Active pelvic infection
- Sexually transmitted disease
- Pregnancy
- Confirmed or suspected intrauterine device
- Uterine perforation

WARNINGS

None known

PRECAUTIONS

This device is intended for use by physicians trained and experienced in intrauterine insemination techniques. Standard techniques should be employed.

INSTRUCTIONS FOR USE

NOTE: To reduce risk of perforation, do not force the catheter if resistance is felt during insertion.

1. Place the patient in the dorsal lithotomy position with her feet in stirrups.
2. Position a vaginal speculum within the vagina so the external cervical os can be visualized during catheter insertion.
3. Cleanse the cervix with an appropriate cleansing solution.
4. Affix the fitting of the catheter to a syringe preloaded with washed sperm solution.
5. Guide the catheter through the cervix and into the uterine cavity. **NOTE:** Select catheters may have positioners or ink marks to facilitate placement.
6. Instill sperm into the uterus.
7. Remove the catheter from the patient.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not

use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd