- EN**  
3 **Entuit® Start** Initial Placement  
Gastrostomy Set  
Instructions for Use
- DA**  
4 **Entuit® Start** gastrotomisæt til  
initial anlæggelse  
Brugsanvisning
- DE**  
5 **Entuit® Start** Gastrostomie-Set für die  
erstmalige Platzierung  
Gebrauchsanweisung
- EL**  
7 ΣΕΤ γαστροστομίας αρχικής τοποθέτησης  
**Entuit® Start**  
Οδηγίες χρήσης
- ES**  
8 Equipo de gastrostomía para colocación  
inicial **Entuit® Start**  
Instrucciones de uso
- FR**  
9 Set de gastrostomie pour mise en place  
initiale **Entuit® Start**  
Mode d'emploi
- IT**  
11 Set per il posizionamento iniziale di cateteri  
per gastrostomia **Entuit® Start**  
Istruzioni per l'uso
- NL**  
12 **Entuit® Start** gastrostomieset voor  
eerste plaatsing  
Gebruiksaanwijzing
- PT**  
14 Conjunto de gastrostomia para colocação  
inicial **Entuit® Start**  
Instruções de utilização
- SV**  
15 **Entuit® Start** gastrostomiset för  
initial placering  
Bruksanvisning

## ENTUIT® START INITIAL PLACEMENT GASTROSTOMY SET

**CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).**

### DESCRIPTION

The Entuit Start Initial Placement Gastrostomy Sets consist of multiple dilators, one or more Peel-Away sheath introducer(s) with dilator(s), and an 0.038", 100 cm long wire guide.

### INTENDED USE

The Entuit Start Initial Placement Gastrostomy Set is intended to assist the physician with the initial placement of feeding tubes while performing a gastrostomy in adult patients only.

### CONTRAINDICATIONS

Contraindications include, but are not limited to:

- Ascites
- Colonic interposition
- Portal hypertension
- Gastric varices
- Peritonitis
- Morbid obesity

### WARNINGS

Aspiration pneumonia may result from feeding tube placement.

### PRECAUTIONS

- Imaging guidance is recommended during use of this product.
- This product is intended for use by physicians trained and experienced in percutaneous diagnostic and interventional techniques. Standard techniques should be employed.
- Prior to use, ensure that the gastrostomy device to be inserted will fit through the lumen of the Peel-Away sheath in the set.

### PRODUCT RECOMMENDATIONS

IPSG Size	Peel-Away Introducer	Recommended Gastrostomy Tube
S	PLVW-16.0-38	12 Fr, 14 Fr
M	PLVW-18.0-38	16 Fr
M	PLVW-20.0-38	18 Fr
L	PLVW-22.0-38	20 Fr
L	PLVW-26.0-38	22 Fr, 24 Fr

### INSTRUCTIONS FOR USE

**NOTE:** Prior to using the components of this set, it is necessary to place gastropexy securement devices and gain percutaneous access to the stomach through the abdominal wall.

1. Under fluoroscopic or endoscopic visualization, insert and advance the 0.038" 100 cm wire guide through the percutaneous access needle cannula. Advance the wire guide far enough to ensure the flexible portion of the wire guide is within the stomach, well past the insertion tract.
2. While maintaining wire guide position, withdraw and remove the needle cannula.
3. Use a scalpel to create a dermatotomy alongside the wire guide at the insertion site, extending through the subcutaneous tissue and fascia.
4. Activate the hydrophilic coating on the dilators by wiping them with a gauze sponge soaked with saline or sterile water. Keep wet during use.
5. While maintaining wire guide position, advance the smallest diameter dilator over the wire and across the tract. Advance the dilator until the full diameter of the dilator has passed through the tract.

**NOTE:** Do not advance the tip of the dilator past the distal end of the wire guide.

6. While maintaining wire guide position, remove the dilator.
7. Repeat step 5 with the remaining dilators, progressing from the smallest diameter to the largest. **NOTE:** When using the IPSG-L set, the 24 French dilator is optional when placing the 22 French sheath.
8. Once the final dilator is removed, insert the appropriately sized Peel-Away introducer over the wire guide, while maintaining wire guide position. **NOTE:** See Product Recommendations section for recommended gastrostomy tube to be used.
9. Confirm endoscopically or fluoroscopically that the distal end of the Peel-Away sheath is through the percutaneous tract and positioned within the stomach.
10. While maintaining position of the Peel-Away sheath, remove the dilator and wire guide.

- Place the gastrostomy device through the Peel-Away sheath and position the device within the stomach.
- While maintaining position of the gastrostomy device, grasp the arms of the Peel-Away sheath and separate the sheath material while withdrawing the material.
- Activate the gastrostomy device's securement mechanism.
- Continue with the procedure per instructions of the gastrostomy device being utilized.

## HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

## REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.