COOK

MEDICAL

EN

3

Soules Intrauterine Insemination Catheter

Instructions for Use

CS

4

Katetr pro intrauterinní inseminaci Soules

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DA

5

Soules intrauterint inseminationskateter

Brugsanvisning

DE

6

Soules intrauteriner Inseminationskatheter

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EL

7

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T _ I U I O _ R E V 2

SOULES INTRAUTERINE INSEMINATION CATHETER

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

Human Sperm Survival Assay (HSSA) $\geq$ 70% motility after 24 hours, USP Endotoxin Assay (LAL) $\leq$ 20 EU per device.

DEVICE DESCRIPTION

The Soules Intrauterine Insemination Catheter is a polyethylene, non-radiopaque catheter with an adjustable silicone reference marker.

INTENDED USE

The Soules Intrauterine Insemination Catheter is intended to access the uterine cavity for saline infusion sonohysterography or for the introduction of washed spermatozoa into the uterine cavity.

CONTRAINDICATIONS

Do not use this device in the presence of:

- Active pelvic infection
- Sexually transmitted disease
- Pregnancy
- Confirmed or suspected intrauterine device
- Uterine perforation

WARNINGS

None known

PRECAUTIONS

This device is intended for use by a physician or properly licensed practitioners trained and experienced in diagnostic and interventional techniques. Standard techniques for SHG procedures should be employed. The IUI procedure should only be performed by physicians trained in intrauterine insemination techniques.

INSTRUCTIONS FOR USE

Sonohysterography (SHG)

1. Place the patient in dorsal lithotomy position with her feet in stirrups.
2. Fill the instillation syringe with sterile saline solution and eliminate air from the syringe.
3. Prime the instillation lumen of the catheter with saline.
4. Position a vaginal speculum within the vagina so the external cervical os can be visualized during catheter insertion.
5. Cleanse the cervix with an appropriate cleansing solution.
6. Insert the catheter tip through cervix and into the uterus. **NOTE:** Depending on the patient, you may choose to use a cervical dilator or tenaculum to facilitate catheter insertion. **NOTE:** The white reference band is positioned 7 cm from the distal tip of the catheter and may be adjusted to assist in catheter placement. **NOTE:** Refrain from forcing the catheter if resistance is felt.
7. Carefully remove the speculum.
8. Insert the transvaginal probe.
9. Inject sufficient saline through the catheter into the uterus.
10. Perform the ultrasound procedure.
11. Remove the catheter from the patient when the examination is complete.

IUI Catheters

1. Place the patient in dorsal lithotomy position with her feet in stirrups.
2. Position a vaginal speculum within the vagina so the external cervical os can be visualized during catheter insertion.
3. Cleanse the cervix with an appropriate cleansing solution.
4. Affix the fitting of catheter to a syringe and load the catheter with washed sperm solution.
5. Guide the catheter through the cervix and into the uterus. **NOTE:** Refrain from forcing the catheter if resistance is felt.
6. Instill sperm into the uterus.
7. Remove the catheter from the patient.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.