EN 3	Goldstein Sonohysterography Catheter Instructions for Use
CS 4	Goldsteinův sonohysterografický katetr Návod k použití
DA 5	Goldstein sonohysterografikateter Brugsanvisning
DE 6	Goldstein Sonohysterographie-Katheter Gebrauchsanweisung
EL 7	Καθετήρας υπερηχοϋστερογραφίας Goldstein Οδηγίες χρήσης
ES 9	Goldstein Sonohysterography Catheter Instrucciones de uso
FR 10	Sonde de Goldstein pour hystérosonographie Mode d'emploi
HU 11	Goldstein szonohiszterográfiás katéter Használati utasítás
IT 12	Catetere per sonoisterografia Goldstein Istruzioni per l'uso
NL 14	Goldstein watercontrastechografiekateter Gebruiksaanwijzing
NO 15	Goldstein-kateter for sonohysterografi Bruksanvisning
PL 16	Cewnik Goldsteina do sonohisterografii Instrukcja użycia
PT 17	Cateter Goldstein para sono-histerografia Instruções de utilização
SV 18	Goldstein sonohysterografikateter Bruksanvisning

GOLDSTEIN SONOHYSTEROGRAPHY CATHETER

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DESCRIPTION

The Goldstein Sonohysterography Catheter has an adjustable acorn positioner and an ink band that provides a reference mark for positioning.

INTENDED USE/INDICATION FOR USE

The Goldstein Sonohysterography Catheter is used to access the uterine cavity for the delivery of saline for sonohysterography (SHG).

CONTRAINDICATIONS

This device should not be used in the presence of:

- Hemorrhage
- Active pelvic infection
- Sexually transmitted disease
- Pregnancy

WARNINGS

To reduce the risk of perforation, if any resistance is felt while inserting the catheter, do not force the catheter against the resistance.

PRECAUTIONS

This procedure should only be performed by a physician or properly licensed practitioner trained in diagnostic and interventional techniques. Standard techniques for SHG procedures should be employed.

POTENTIAL ADVERSE EVENTS

- Discomfort, pain/cramping

INSTRUCTIONS FOR USE

1. Place patient in dorsal lithotomy position.
2. Fill the instillation syringe with saline solution and eliminate air from the syringe.
3. Prime the instillation lumen of the catheter with saline.
4. Position a vaginal speculum within the vagina so the external cervical os can be visualized during catheter insertion.
5. Cleanse the cervix with an appropriate cleansing solution.
6. With cervix exposed, insert catheter into uterus using packing forceps. **NOTE:** Reference mark on catheter may be used to aid in positioning, if present.
7. Advance positioner into cervix using forceps.
8. Carefully remove the speculum.
9. Insert the transvaginal probe.
10. Inject sufficient saline through the catheter into the uterus.
11. Perform ultrasound procedure.
12. Remove the catheter from patient when examination is complete and discard.

NOTE: Upon removal of catheter, ensure that positioner is present. If still lodged in patient cervix, remove using forceps.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK 2020
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd