**COOK**

**MEDICAL**

**EN**  
**3**

**Margolin HSG Cannula**

Instructions for Use

**CS**  
**4**

**Kanyla Margolin HSG**

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**DA**  
**6**

**Margolin HSG-kanyle**

Brugsanvisning

**DE**  
**7**

**Margolin HSG-Kanüle**

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**EL**  
**9**

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T \_ J - M H S G \_ R E V 1

## MARGOLIN HSG CANNULA

**CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).**

### DEVICE DESCRIPTION

The Margolin HSG Cannula is a catheter with acorn tip and stylet.

### INTENDED USE/INDICATION FOR USE

The Margolin HSG Cannula is for the delivery of contrast media or saline into the uterine cavity during Hysterosalpingography (HSG) or Sonohysterography (SHG) for examination of the uterus and fallopian tubes. When used for HSG, the Margolin HSG Cannula can be used for evaluation of tubal patency.

### CONTRAINDICATIONS

This device should not be used in the presence of:

- Hemorrhage
- Active pelvic infection
- Sexually transmitted disease
- Pregnancy
- Confirmed or suspected intrauterine device

### WARNINGS

To reduce the risk of perforation, if any resistance is felt while inserting the catheter, do not force the catheter against the resistance.

### PRECAUTIONS

- This device is intended for use by a physician or properly licensed practitioner trained in diagnostic and interventional techniques. Standard techniques for SIS/HSG procedures should be employed.
- Use of the catheter in postpartum patients, patients with recent cesarean section, or patients with a recent uterine perforation should be determined at the physician's discretion.

### POTENTIAL ADVERSE EVENTS

- Hypersensitivity to contrast media
- Discomfort, pain/cramping
- Infection

### INSTRUCTIONS FOR USE

**NOTE:** To reduce risk of perforation, do not force the catheter if resistance is felt during insertion.

#### Hysterosalpingography

1. Place patient in dorsal lithotomy position.
2. Fill the instillation syringe with contrast medium and eliminate air from the syringe.
3. Position a vaginal speculum within the vagina so the external cervical os can be visualized during catheter insertion.
4. Cleanse the cervix with an appropriate cleansing solution.
5. Insert the catheter tip through cervix and into the uterus and remove stylet.
6. Inject sufficient contrast medium through the catheter into the uterus.
7. Perform the fluoroscopy procedure.

8. Remove the catheter from the patient when the examination is complete and discard.

### **Sonohysterogram**

1. Place patient in dorsal lithotomy position.
2. Fill the instillation syringe with saline solution and eliminate air from the syringe.
3. Position a vaginal speculum within the vagina so the external cervical os can be visualized during catheter insertion.
4. Cleanse the cervix with an appropriate cleansing solution.
5. Insert the catheter tip through cervix and into the uterus and remove stylet.
6. Inject sufficient saline through the catheter into the uterus.
7. Perform ultrasound procedure.
8. Remove the catheter from the patient when the examination is complete and discard.

### **HOW SUPPLIED**

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

### **REFERENCES**

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.