- EN**  
**5**  
**Flexipet™ Reusable Handle**  
Instructions for Use
- CS**  
**6**  
**Rukojeť Flexipet™ k opakovanému použití**  
Návod k použití
- DA**  
**7**  
**Flexipet™ genanvendeligt håndtag**  
Brugsanvisning
- DE**  
**9**  
**Flexipet™ wiederverwendbarer Griff**  
Gebrauchsanweisung
- EL**  
**10**  
**Επαναχρησιμοποιήσιμη λαβή Flexipet™**  
Οδηγίες χρήσης
- ES**  
**12**  
**Mango reutilizable Flexipet™**  
Instrucciones de uso
- FR**  
**13**  
**Poignée réutilisable Flexipet™**  
Mode d'emploi
- HU**  
**14**  
**Flexipet™ újrafelhasználható fogantyú**  
Használati utasítás
- IT**  
**16**  
**Impugnatura riutilizzabile Flexipet™**  
Istruzioni per l'uso
- NL**  
**17**  
**Flexipet™ herbruikbare handgreep**  
Gebruiksaanwijzing
- NO**  
**18**  
**Flexipet™ gjenbrukbart håndtak**  
Bruksanvisning
- PL**  
**20**  
**Uchwyt wielokrotnego użytku Flexipet™**  
Instrukcja użycia
- PT**  
**21**  
**Punho reutilizável Flexipet™**  
Instruções de utilização
- SV**  
**23**  
**Flexipet™ återanvändbart handtag**  
Bruksanvisning

T \_ K - M P H \_ R E V 1

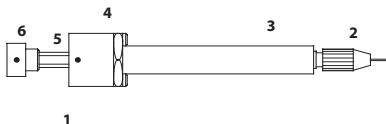
Fig. 1

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Fig. 5

## FLEXIPET™ REUSABLE HANDLE

**CAUTION:** U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

### INTENDED USE

Used with Flexipet denuding and manipulation pipettes.

### CONTRAINDICATIONS

None known

### WARNINGS

None known

### PRECAUTIONS

The potential effects of phthalates on pregnant/nursing women or children have not been fully characterized and there may be concern for reproductive and developmental effects.

### INSTRUCTIONS FOR USE

#### Adjusting Aspiration Volume (Fig.2)

- Rotate the plunger clockwise to decrease aspiration volume.
- Rotate the plunger counterclockwise to increase aspiration volume.

| Aspiration Volume Adjustment Chart |        |
|------------------------------------|--------|
| Total Capacity                     | 3.0µl  |
| 360 Rotation                       | 0.2µl  |
| 180 Rotation                       | 0.1µl  |
| 90 Rotation                        | 0.05µl |

#### Installing Flexipets

1. Loosen the collet by turning counterclockwise at least one full rotation.
2. Slide a Flexipet onto the plunger and gently advance into the collet until firmly seated. Tighten the collet.

**IMPORTANT:** Over-tightening the collet can compress the o-rings and restrict plunger movement. If the plunger fails to move freely, loosen the collet until the desired movement is achieved.

#### Aspiration Technique

1. Depress the plunger until the finger pad contacts the spring housing, immerse the tip into the culture media, and slowly release the plunger.
2. Expel media by depressing the plunger until the finger pad is completely depressed [blow out] (**Fig. 3**). To avoid air bubbles, stop the plunger before it is completely depressed. The small amount of medium remaining in the extreme tip can be touched off.

## O-Rings Replacement

O-rings should be replaced if leakage occurs or if the Flexipet cannot be properly secured to the handle.

**IMPORTANT:** To ensure proper operation, the o-rings must be installed as depicted in **Fig. 4** and **Fig. 5**, with the smaller aperture o-ring (1) installed first. Also, the second o-ring (2) must be installed with the larger aperture positioned toward the collet so the Flexipet will fully insert into the o-ring (**Fig. 5**).

A Flexipet o-ring replacement set may be ordered (K-FRO-1000), consisting of one small aperture o-ring, one large aperture o-ring, spacer and collet.

## HOW SUPPLIED

Supplied non-sterile in peel-open packages. Reusable. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

## REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.