Koyle Diaper Stent

Plenkový stent Koyle

Koyle blestent

Koyle Windel-Stent

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EN
3
CS
3
DA
4
DE
4
EL
5
ES
6
FR
6
HU
7
IT
8
NL
8
NO
9
PL
10
PT
10
SV
11
ZH
12

KOYLE DIAPER STENT

Used for stenting the urethra following hypospadias or epispadias repair to provide postoperative drainage of the bladder. The externalized portion of the stent is of sufficient length to allow drainage into a second or outside diaper, keeping the inside diaper dry during the healing process. Intended for one-time use.

DESCRIPTION

- Urethral stent
- Introducer (*optional*)

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CONTRAINDICATIONS

There are no known contraindications.

SUGGESTED INSTRUCTIONS FOR USING KOYLE DIAPER STENT

1. Measure the urethral length from the bladder to the urethral meatus utilizing a feeding tube.
2. Using the feeding tube as a guide, cut the urethral segment of the stent to the same length.
3. Advance the stent into the urethra. **NOTE: Utilize the optional introducer to assist in difficult placement. Remove the introducer following placement.**
4. Using absorbable suture, anchor the stent to the glans meatus at the funneled area.
5. Apply appropriate dressing and place diaper on patient, leaving the distal segment outside the diaper.
6. Place an additional, larger outside diaper over the inner and route the distal drainage segment into the outer diaper.

NOTE: The stent is left in place four (4) weeks. During the healing process the sutures are absorbed and the stent should be expelled spontaneously during micturition.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.