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T _ M C S S _ R E V O

MALECOT NEPHROSTOMY CATHETER/STENT SET

Used as a nephrostomy drainage catheter and ureteral stent. The Malecot catheter is placed percutaneously through an existing nephrostomy tract. Intended for one-time use.

SET CONTAINS *(set components may vary)*

- Radiopaque Malecot catheter/stent and two (2) flexible stylets
- Retention disc with pull tie
- Connecting tube

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CONTRAINDICATIONS

There are no known contraindications.

SUGGESTED INSTRUCTIONS FOR USING MALECOT NEPHROSTOMY CATHETER/STENT SET

1. Perform the standard techniques for establishing a nephrostomy tract utilizing fluoroscopic control, leaving a 145 cm wire guide in the collecting system. Ensure fluoroscopically that the wire guide travels the length of the ureter and is well into the bladder.
2. To facilitate passage of the nephrostomy catheter, dilate the musculofascial tract by progressing from the smallest to the largest dilator. If replacing a Malecot catheter for another, skip this step and proceed with step number 3.
3. Place the blunt stylet inside the Malecot catheter and secure its position with the Luer lock to straighten the Malecot wings.
4. Pass the stent end of the Malecot catheter over the external end of the wire guide; gradually advance the catheter end well into the collecting system and place the stent segment into the ureter the desired length. Confirm the position fluoroscopically. After releasing the Luer lock in order to open the Malecot wings, carefully remove the blunt stylet. **NOTE: Care must be exercised to avoid withdrawal of the wire guide itself.**
5. While the shaft of the Malecot catheter is held securely in position with one hand, the wire guide is withdrawn with the other. When the appropriate position of the Malecot is assured by fluoroscopic visualization, remove the wire guide.
6. Secure the catheter to the skin with the retention disc provided. Tape or suture the retention disc to the skin; secure the catheter to the disc with the plastic pull tie. A dressing may be applied. Use the connecting tube to connect the Malecot catheter to a drainage bag or leg bag.

CAUTION: The indwelling time should not exceed four (4) weeks. Evaluation and replacement beyond 4 weeks must be considered.

NOTE: The individually packaged, flexible catheter stylet should be reserved for straightening the Malecot catheter prior to removal at a later date.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.