The Graduate® Measuring Wire Guides

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ZH
10

T _ M G _ R E V . 1

THE GRADUATE® MEASURING WIRE GUIDES

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Graduate Measuring Wire Guide is a TFE-coated wire guide with gold, radiopaque markers delineating 25 cm in length. The four proximal markers are spaced at 5 cm intervals, and the six distal markers are spaced at 1 cm intervals.

INTENDED USE

The Graduate Measuring Wire Guide is intended to determine accurate sizing of vessel lumen prior to angioplasty, embolization and other interventional procedures. The product is intended for use by physicians trained and experienced in diagnostic and interventional techniques. Standard techniques for placement of wire guides should be employed.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

- Manipulation of wire guides requires appropriate imaging control.
- When using wire guide with another device(s), the endhole size and length of the device(s) must be taken into consideration to ensure proper fit between wire guide and device(s).
- Withdrawal or manipulation of wire guide through a needle tip may result in breakage.
- Altering tip configuration or curve may result in damage to the wire guide.

INSTRUCTIONS FOR USE

1. Use standard procedure for accessing the target vessel.
NOTE: If resistance is met, stop manipulating wire guide and determine cause of resistance via appropriate imaging.
2. Advance the wire guide to the desired location.
3. Use fluoroscopy to visualize the gold markers spaced at 5 cm intervals to perform the necessary sizing.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.