**EN**  
3

**NCompass® Nitinol Stone Extractors**

Instructions for Use

**DA**  
4

**NCompass® stenekstraktorer af nitinol**

Brugsanvisning

**DE**  
5

**NCompass® Nitinol-Steinentferner**

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**EL**  
7

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T \_ N C N S E \_ R E V 1

## NCOMPASS® NITINOL STONE EXTRACTORS

**CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).**

### DEVICE DESCRIPTION

The NCompass Nitinol Stone Extractor is a nitinol basket with an outer sheath diameter of 2.4 French and a 1.5 cm basket diameter when fully expanded. The tipped basket is used under fluoroscopic guidance; the tiptless basket is used for procedures utilizing a choledochoscope.

### INTENDED USE

The NCompass Nitinol Stone Extractors are intended for extraction of stones or debris during biliary surgical procedures.

### CONTRAINDICATIONS

This device is not intended for use within Endoscopic Retrograde Cholangio-Pancreatography (ERCP) procedures.

### WARNINGS

- This device can conduct electrical current. Avoid contact with any electrified instrument.
- The device is designed for the removal of stones and debris no larger than 0.8 cm in diameter.

### PRECAUTIONS

- Refer to the package label for the minimum channel size required for this device.
- Enclose the basket in the sheath before removing the device from the holder.
- Pulling on the sheath while advancing or retracting the basket may damage the device, rendering it inoperable.
- If resistance is encountered while attempting to remove a stone or other debris, release the object. Excessive force could result in device damage, such as the inability to open/close device and basket separation.
- The tipped NCompass 3 wire Nitinol Stone Extractor is designed to be used under fluoroscopic guidance. The use of the tipped extractor with a scope could damage the scope.
- This product is intended for use by physicians trained and experienced in laparoscopic common duct exploration (LCDE) and/or laparotomy with open common bile duct exploration (OCDE). Standard techniques should be employed.
- The potential effects of phthalates on pregnant/nursing women or children have not been fully characterized and there may be concern for reproductive and developmental effects.

### POTENTIAL ADVERSE EVENTS

- Bleeding
- Ductal perforation
- Stone impaction

### INSTRUCTIONS FOR USE

**NOTE:** This device is designed to remove stones no larger than 0.8 cm in diameter.

1. Remove the stone extractor from its outer package and move the thumb tab on the handle to the "Close" position to close the basket.
2. Insert the stone extractor either through the working channel of the choledochoscope or a lumen of an ancillary device used to access the bile duct (for example, a cholangiography catheter or dilation balloon with a working channel of 2.4 French or greater).

**NOTE:** The provided Tuohy-Borst adapter, featuring a distal Luer lock connector, may be attached to the proximal end of the ancillary device prior to the insertion of the stone extractor, if desired.

3. Advance the stone extractor into the duct and position the distal end beyond the stone or debris.
 

**NOTE:** Under fluoroscopic monitoring, the tip of the tipped stone extractor (C-NTSE-2.4-115-NC3) may be visualized.
4. After confirming that the extractor is positioned alongside or beyond the stone/debris, open the basket by sliding the thumb tab to the "Open" position.
5. Maneuver and manipulate the open basket to capture the stone or debris.
6. Once captured, slowly transition the thumb tab toward the "Close" position to secure the stone or debris.
7. Maintain pressure on the thumb tab of the handle to hold the stone or debris in place. Slowly retract the stone extractor and the ancillary device (or scope if utilized) together to facilitate the immediate removal of object from the tract.

**NOTE:** The provided Tuohy-Borst adapter may be tightened once the stone is captured to secure the position of the stone extractor during removal. This adapter will not lock the basket on the stone or debris, but will secure the position of the sheath relative to the ancillary device.

**NOTE:** Do not use excessive force on the handle. Doing so may result in damage to the device, including but not limited to, basket separation or inability to open/close device.

8. If multiple passes are needed to collect multiple stones, repeat steps 2-7 to remove each stone individually. To avoid impaction, begin with the stone most proximal to the operator.

## HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

## REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for more information.

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Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd