- EN 3** N-Circle Nitinol Hysteroscopic Polyp Snare
Instructions For Use
- CS 4** Nitinolové hysteroskopické očko na polypy N-Circle
Návod k použití
- DA 5** N-Circle hysteroskopisk nitinolpolypslynge
Brugsanvisning
- DE 6** N-Circle hysteroskopische Polypen-Drahtschlinge aus Nitinol
Gebrauchsanweisung
- EL 7** Υστεροσκοπικός βρόχος για πολύποδες από nitinol N-Circle
Οδηγίες χρήσης
- ES 8** Asa de nitinol para polipectomía histeroscópica N-Circle
Instrucciones de uso
- FR 9** Anse à polypectomie hystéoscopique en nitinol N-Circle
Mode d'emploi
- HU 10** N-Circle nitinol hiszteroszkópiás poliphurok
Használati utasítás
- IT 11** Ansa per polipectomia isteroscopica in nitinolo N-Circle
Istruzioni per l'uso
- NL 12** N-Circle hysteroscopische poliepsnaar van nitinol
Gebruiksaanwijzing
- NO 13** N-Circle hysteroskopisk polypsslynge av nitinol
Bruksanvisning
- PL 14** Histeroskopowa pętla nitinolowa N-Circle do usuwania polipów
Instrukcja użycia
- PT 15** Ansa histeroscópica para pólipos em nitinol N-Circle
Instruções de utilização
- SV 16** N-Circle hysteroskopisk polypsnara av nitinol
Bruksanvisning
- ZH 17** N-Circle镍钛合金宫腔镜息肉勒除器
使用说明

N-CIRCLE® NITINOL HYSTEROSCOPIC POLYP SNARE

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

- Hysteroscopic Polyp Snare with 4-wire nitinol snare

INTENDED USE

Used for manipulation and removal of uterine polyps under direct vision. The nitinol memory wire opens easily in the uterine cavity allowing capture and removal of polyps. The small diameter sheath permits use with rigid or flexible hysteroscopes.

CONTRAINDICATIONS

This device should not be used on a patient with an active vaginal or intrauterine infection, a sexually transmitted disease, a recent uterine perforation, a recent cesarean section, who is currently pregnant, or if the patient currently has, or suspects she may have an intrauterine device within her uterus.

WARNINGS

- This device is not intended to be used concomitantly with an electrocautery power source. This device removes polyps by mechanical force alone, and should not be attached to any power source.

PRECAUTIONS

- As some polyps have vascular sources and attachments, there may be noticeable bleeding at the site of polyp removal. Appropriate care should be taken to treat excessive bleeding including but not limited to, electrocautery coagulation or tamponade of bleeding site.
- Maximum polyp size is 2cm diameter.
- Described snare techniques are most effective when used to treat pedunculated polyps.
- Excessive force could damage device.

INSTRUCTIONS FOR USING USING N-CIRCLE® NITINOL HYSTEROSCOPIC POLYP SNARE

1. Retract the snare into the sheath.
2. Insert the sheath and snare assembly through the accessory channel of the hysteroscope until the tip is visualized in the uterine cavity.
3. Open snare near the polyp/tumor and maneuver snare until polyp/tumor is in snare basket.
4. Draw the snare down around the polyp/tumor until slight resistance is felt.
5. Apply gentle traction against polyp to remove the polyp from the uterine wall.
6. Once detached from the uterine wall, maintain purchase of the polyp with the snare basket and retract scope and snare as one from the uterine cavity to facilitate immediate removal of the polyp body.
7. Re-insert scope and identify any remaining polyp fragments. Those smaller remaining fragments should be removed with appropriate forceps or graspers.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.