EN
3

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Bruksanvisning

PERCUTANEOUS PIGTAIL NEPHROSTOMY SET WITH MITTY-POLLACK NEEDLE SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

- Mitty-Pollack needle set
- Wire guide with 3 mm Safe-T-J® tip
- Radiopaque dilators
- Radiopaque pigtail catheter
- Retention disc with pull tie
- Connecting tube

NOTE: Set components may vary.

INTENDED USE

Used for percutaneous placement of a pigtail catheter in the renal pelvis for nephrostomy drainage.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

- Periodic evaluation is advised; catheter must not remain indwelling more than four months.
- The potential effects of phthalates on pregnant/nursing women or children have not been fully characterized and there may be concern for reproductive and developmental effects.

INSTRUCTIONS FOR USE

1. By preliminary plain film, I.V.P., retrograde pyelogram, ultrasound or CAT scan, localize the kidney to be drained. A prone or slightly oblique position is preferred. Prepare and drape the flank in the usual fashion. Under fluoroscopic control, identify and anesthetize the skin site overlying the collecting system.
2. Rotate the patient into a more lateral position, with the posterior axillary line of the ipsilateral flank in the superior position. Pass the assembled Mitty-Pollack needle vertically through a small incision in the skin (made with a scalpel blade); insert the needle into the appropriate part of the pyelocalyceal system. Free flow of fluid from the needle after removing the obturator confirms a satisfactory location for the tip of the 22 gage needle cannula.
3. While holding the inner 22 gage needle in place, unlock the outer 18 gage needle from the hub and advance the outer needle over the inner needle to the desired location. Once in position, remove the inner 22 gage needle.
4. Pass the flexible end of the Safe-T-J® wire guide through the hub of the 18 gage needle cannula; insert the wire guide three more inches into the collecting system. Confirm the position fluoroscopically. Maintain the position of the wire guide as the needle cannula is withdrawn over it. **NOTE:** Care must be exercised to avoid withdrawal of the wire guide itself.
5. To facilitate passage of the nephrostomy catheter, dilate the musculofascial tract by progressing from the smallest to the largest dilator provided.
6. Pass the pigtail end of the catheter over the external end of the wire guide; gradually advance the pigtail end well into the collecting system. Confirm the position fluoroscopically. While holding the shaft of the pigtail catheter securely in position with one hand, withdraw the wire guide with the other.

7. Tape or suture the retention disc to the skin; secure the catheter to the disc with the plastic tie. A dressing may be applied. Use the connecting tube to connect the pigtail catheter to a drainage bag or leg bag. The one-way stopcock is useful for testing the drainage capability of the collecting system, if desired.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

A. B. Rutner, I. S. Fucilla: "Percutaneous Pigtail Nephrostomy," *Urology*, 14 (1979), 337.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.