EN
3

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T _ N T P _ R E V O

NTRAP® Stone Entrapment and Extraction Device

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

Set contains (*set components may vary*)

- Stone Extractor
- Unidex Handle

INTENDED USE

Used as an endoscopic entrapment and extraction device for calculi and other foreign bodies in the urinary tract, and to minimize stone migration during laser, ultrasonic, electrohydraulic or pneumatic lithotripsy.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

- Users should be familiar with and experienced in urological endoscopic surgery.
- Assess calculi or other foreign bodies prior to instrument deployment to ensure that object is not too large to be removed through the anatomy.
- If resistance is encountered while attempting to remove calculi or other foreign bodies, release the object. Do not exert excessive force on the device.
- Due to the asymmetric nature of the NTRAP, do not torque or rotate the device in vivo.
- When used in conjunction with holmium: YAG laser, do not fire the laser directly upon the device.
- This device is conductive. Avoid contact with any electrified instrument.
- Enclose device in sheath before removing from tray/holder.

POTENTIAL ADVERSE EVENTS

The following complications are possible with the use of stone extraction devices.

- Tissue perforation
- Tissue evulsion
- Edema
- Hemorrhage

INSTRUCTIONS FOR USE

1. Using aseptic technique, remove the NTRAP from its outer package and place into the sterile field. The NTRAP operating handle is attached to facilitate opening or closing.
2. Before insertion, retract the NTRAP basket into the sheath.
3. Using direct vision or fluoroscopic control, advance the NTRAP sheath beyond the stone or foreign object prior to deployment.
4. Deploy the NTRAP by advancing the handle forward while holding the sheath in position. The device may now be used to minimize stone migration during laser, ultrasonic, electrohydraulic or pneumatic lithotripsy.

NOTE: When using laser or electrohydraulic lithotripsy, a gap of at least 1 cm should be maintained between the stone and the NTRAP to prevent device damage.

NOTE: The NTRAP, in the open position, may be used as an extraction device to sweep the ureter, collecting stone fragments into the bladder. The operating handle may be detached to facilitate scope removal.

5. Upon completion of the procedure, close the NTRAP basket by pushing the sheath forward while holding the operating handle in position.
6. It may be necessary to gently advance the basket to release any remaining stone fragments prior to closing.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.