- EN**
3 **NCircle[®], NCompass[®], and NForce[®] Stone Extractors**
Instructions for Use
- CS**
4 **Extraktory kaménků NCircle[®], NCompass[®] a NForce[®]**
Návod k použití
- DA**
5 **NCircle[®]-, NCompass[®]- og NForce[®]- stenekstraktorer**
Brugsanvisning
- DE**
6 **NCircle[®], NCompass[®] und NForce[®] Steinextraktoren**
Gebrauchsanweisung
- EL**
7 **Εξαγωγείς λίθων NCircle[®], NCompass[®] και NForce[®]**
Οδηγίες χρήσης
- ES**
8 **Extractores de cálculos NCircle[®], NCompass[®] y NForce[®]**
Instrucciones de uso
- FR**
9 **Extracteurs de calculs NCircle[®], NCompass[®] et NForce[®]**
Mode d'emploi
- HU**
10 **NCircle[®], NCompass[®] és NForce[®] kőeltávolítók**
Használati utasítás
- IT**
11 **Estrattori di calcoli NCircle[®], NCompass[®] e NForce[®]**
Istruzioni per l'uso
- NL**
12 **NCircle[®], NCompass[®] en NForce[®] steenextractors**
Gebruiksaanwijzing
- NO**
13 **NCircle[®], NCompass[®] og NForce[®] stenekstraktorer**
Bruksanvisning
- PL**
14 **Ekstraktory żłogów NCircle[®], NCompass[®] i NForce[®]**
Instrukcja użycia
- PT**
15 **Extractores de cálculos NCircle[®], NCompass[®] e NForce[®]**
Instruções de utilização
- SV**
16 **NCircle[®], NCompass[®] och NForce[®] stenutdragare**
Bruksanvisning

NCIRCLE®, NCOMPASS®, AND NFORCE® STONE EXTRACTORS

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The NCircle, NCompass, and NForce Stone Extractors are 3- or 4-wire nitinol baskets available in a range of French sizes, lengths, and basket diameters.

INTENDED USE

The NCircle, NCompass, and NForce Stone Extractors are intended for stone manipulation and removal in the urinary tract.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

- These products are intended for use by physicians trained and experienced in urological procedures. Standard urological techniques should be employed.
- This device is conductive. Avoid contact with any electrified instrument.
- Enclose the device in the sheath before removing from the tray/holder.
- Do not use excessive force to manipulate this device. Damage to the device may occur.

INSTRUCTIONS FOR USE

1. Remove the stone extractor from its outer package and move the thumb lever on the handle to the "Close" position to close the basket.
2. Place the instrument insertion adapter (if included) securely into the endoscopic cap on the scope.
3. With the insertion adapter in place, carefully insert the instrument into the scope.
4. Under direct visualization, locate the stone and advance the extractor to a location just proximal to the stone. Slide the thumb lever on the handle to the "Open" position to deploy the basket near the stone and maneuver the device until the stone is within the basket.
5. Once correctly positioned, draw the basket down around the stone and slide the thumb lever into the "Close" position.
6. Maintaining position of the stone within the basket, retract the scope and basket.

HOW SUPPLIED

Supplied sterile by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK 2019
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd