COOK
MEDICAL

CE
0123

EN
3

**One-Step Suprapubic
Introducer**

Instructions for Use

CS
4

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DA
6

Ettrins suprapubisk indfører

Brugsanvisning

DE
7

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T - O S I - R E V S

ONE-STEP SUPRAPUBIC INTRODUCER

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner). Please read all instructions before using this device.

DEVICE DESCRIPTION

Set Contains:

- Trocar Stylet
- Dilator
- Peel-Away® Sheath

INTENDED USE

The One-Step Suprapubic Introducer is intended for introducing a drainage catheter suprapubically into the bladder.

CONTRAINDICATIONS

- Carcinoma of the bladder
- Presence of subcutaneous vascular graft in the suprapubic area

WARNINGS

- Sterilized by Ethylene Oxide gas. Do not use if sterile barrier is damaged. The device is intended for one-time use.
- Do not attempt insertion if bladder cannot be located.

PRECAUTIONS

- This device is intended for use by physicians trained and experienced in techniques for suprapubic access. Standard techniques should be employed.
- Ensure patient has a full and palpable bladder for correct bladder position.
- Do not forcefully advance any component during removal or replacement of the catheter. Carefully remove the components if any resistance is encountered.

POTENTIAL ADVERSE EVENTS

- Complications that might occur during this procedure include incorrect location at insertion, resulting in trauma to the surrounding tissue.
- Mild burning at insertion site
- Blood in urine
- Cystitis, urosepsis
- Blocked catheter
- Development of bladder stomas
- Bladder spasm
- Insertion site infection

INSTRUCTIONS FOR USE

1. Using aseptic technique, remove the introducer from its outer packaging and place in the sterile field. Prepare the puncture site in the usual manner. Ensure the patient has a full and palpable bladder to confirm urine return.
2. Inspect the introducer to ensure no damage has occurred during shipment. Defective devices should be returned to Cook.

NOTE: Remove the needle tip protector prior to insertion.

3. Ensure components are assembled into one piece.
4. Use a scalpel to make a small incision at the puncture site. Inject local anesthesia to anesthetize the rectus sheath.
5. Advance the assembled components with one single pass suprapubically through the rectal sheath and into the bladder.
6. Remove the trocar stylet to confirm the device is within the bladder by return of urine flow. If spontaneous urine flow is not established, a syringe may be attached to the dilator for aspiration.

NOTE: Only proceed if a flow of urine is confirmed.

7. Remove the dilator from the Peel-Away Sheath to allow placement of an appropriately sized drainage catheter through the sheath.

CAUTION: The outer diameter of drainage catheters vary from manufacturer to manufacturer. Care should be taken to verify the drainage catheter will pass through the Peel-Away Sheath prior to suprapubic insertion.

8. Insert the drainage catheter as usual through the sheath. Using each side of the Peel-Away Sheath, gently pull back and split the sheath into two parts while leaving the catheter in situ.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

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Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd