EN
3

O'Brien Suprapubic Peel-Away® Access Set

Instructions for Use

CS
4

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T _ O S P A S _ R E V 2

O'BRIEN SUPRAPUBIC PEEL-AWAY® ACCESS SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner). Please read all instructions before using this device.

DEVICE DESCRIPTION

Set Contains:

- Introducing needle
- Stainless steel wire guide
- Peel-Away® sheath

INTENDED USE

The O'Brien Suprapubic Peel-Away® Access Set is intended for introducing a drainage catheter suprapubically into the bladder.

CONTRAINDICATIONS

None known

WARNINGS

Sterilized by Ethylene Oxide gas. Do not use if sterile barrier is damaged. The device is intended for one-time use.

PRECAUTIONS

- This device is intended for use by physicians trained and experienced in techniques for suprapubic access. Standard techniques should be employed.
- Ensure patient has a full and palpable bladder for correct bladder position.
- Do not forcefully advance any component during removal or replacement of the catheter. Carefully remove the components if any resistance is encountered.

POTENTIAL ADVERSE EVENTS

Complications that might occur during this procedure could include incorrect location at insertion, resulting in trauma to the surrounding tissue.

INSTRUCTIONS FOR USE

1. Prepare puncture site in the usual manner. Ensure the patient has a full and palpable bladder to confirm urine return.
2. Advance the introducing needle suprapubically into the bladder. Confirm the needle is within the bladder by return of urine flow. **NOTE:** Only proceed if a flow of urine is confirmed.
3. Advance the wire guide through the introducing needle and into the bladder.
4. Once the wire guide is in place, remove the introducing needle.
5. Appropriately manage pain with local anesthesia.
6. Use a scalpel to make a skin puncture at the midline of the wire guide exit site.
7. If needed, dilate the tract with dilator(s) of choice up to the size of the O'Brien Suprapubic introducer sheath.
8. Introduce the O'Brien Suprapubic introducer sheath over the wire and remove the dilator.
9. Urine flow will confirm that the introducer sheath is positioned in the bladder. Compress the introducer sheath between the thumb and forefinger to stop urine flow. **NOTE:** If the drainage catheter of choice does not require the wire guide for placement, the wire guide can be removed.

10. Introduce the drainage catheter of choice through the introducer sheath into the bladder. Secure the catheter in the usual manner. **CAUTION:** The outer diameter of drainage catheters may vary from manufacturer to manufacturer. Care should be taken to verify the drainage catheter will pass through the Peel-Away sheath prior to suprapubic insertion.
11. Using each side of the Peel-Away sheath, gently pull back and split the sheath into two parts while leaving the catheter *in situ*.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

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Bloomington, IN 47404 U.S.A.