COOK
MEDICAL

CE
0123

EN
3

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T _ P A I S _ R E V 2

PEEL-AWAY INTRODUCER SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

The Peel-Away Introducer Set consists of a coaxial introducer assembly (Peel-Away sheath and dilator). Sets may also contain a wire guide in a size fitted to the endhole of the dilator and may contain an access needle in a gauge that allows the included wire guide to pass through its lumen.

INTENDED USE

The Peel-Away Introducer Set is intended for the percutaneous introduction of balloon, electrode and closed or non-tapered end catheters into central and peripheral vasculature, and for nonvascular use.

CONTRAINDICATIONS

Patient has a known or suspected obstruction in the vessel.

RELATIVE CONTRAINDICATIONS

During gastroenterologic procedures:

- Ascites
- Colonic interposition
- Portal hypertension
- Gastric varices
- Peritonitis
- Morbid obesity

During pulmonologic procedures:

- Severe chronic lung disease (may increase risk of pneumothorax)
- Irradiation to the anterior chest (may result in poor healing)

WARNINGS

- Do not alter this device in any way.
- Do not withdraw the wire guide through a metal needle; the wire guide may shear or unravel.
- Aspiration pneumonia may result from feeding tube placement.

PRECAUTIONS

- The wire guide in the Peel-Away Introducer Set should not be withdrawn through the needle once it has been placed beyond the needle tip.
- This product is intended for use by clinicians trained and experienced in diagnostic and interventional techniques.
- If you encounter resistance during wire guide or catheter manipulation, stop and determine the cause before proceeding further.
NOTE: If you encounter resistance during insertion, do not force the device.
- The maximum diameter of the instrument or catheter to be introduced should be determined to ensure its passage through the introducer.
- When inserting, manipulating, or withdrawing a device through an introducer, always maintain introducer position.

- Pre-clinical data has shown minimal thrombus formation in the absence of anticoagulation which is comparable to other peel-away introducers.

INSTRUCTIONS FOR USE

1. Prep and drape the access site according to hospital protocol.
2. Introduce an access needle into the target anatomy.
3. Introduce a wire guide through the needle and into the target anatomy.
4. Withdraw the needle, leaving the wire guide in place. If desired, enlarge the skin access site with a #11 scalpel blade.
5. Introduce the Peel-Away introducer assembly (sheath and dilator) over the wire guide. With a twisting motion, advance the assembly into the target anatomy.
6. Leaving the sheath in place, remove the dilator and wire guide.

NOTE: In venous placements, to prevent inadvertent air aspiration after removing the wire guide and dilator, place a thumb or finger over the proximal end of the sheath.

7. Introduce the desired catheter/device into the sheath to the desired depth within the target anatomy.
8. Peel the sheath away from the catheter by grasping the two wings of the sheath and pulling outward and upward.

NOTE: Once the sheath is removed, the catheter/device position may need to be adjusted.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

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Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd