COOK
MEDICAL

CE
0123

EN
3

Pediatric Ureteral Catheter

Instructions for Use

CS
4

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DA
5

Pædiatrisk ureterkateter

Brugsanvisning

DE
6

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EL
8

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T _ P E D U C _ R E V 1

PEDIATRIC URETERAL CATHETER

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

INTENDED USE

Pediatric Ureteral Catheters are indicated for access and catheterization of the urinary tract in pediatric patients, including the following applications.

- Delivery of contrast media
- Drainage of fluids from the urinary tract
- Navigation of a tortuous ureter
- Delivery of irrigation fluids to the urinary tract

CONTRAINDICATIONS

This device is contraindicated for use in patients unsuitable for catheterization.

WARNINGS

None known

PRECAUTIONS

- This device is intended for use by physicians trained and experienced in urological techniques. Standard techniques should be employed.
- Avoid bending or kinking the catheter prior to placement. Doing so could damage the integrity of the catheter and result in patient injury.
- Stop catheter manipulation if you encounter resistance while advancing or withdrawing the catheter. Determine the cause of the resistance before proceeding.
- Do not withdraw the catheter while it is deflected in the cystoscope/endoscope.

POTENTIAL ADVERSE EVENTS

Adverse events that may result from ureteral catheter placement include, but are not limited to:

- Extravasation
- Hemorrhage
- Sepsis
- Edema
- Loss of renal function
- Perforation or laceration of the kidney, renal pelvis, ureter, and/or bladder
- Peritonitis
- Urinary tract infection
- Abrasion

INSTRUCTIONS FOR USE

1. Flush the catheter.
2. With a cystoscope in the bladder, locate the ureteral orifice.
3. Under direct vision, advance the catheter through the cystoscope and position the catheter tip within the ureteral orifice.
4. Once the catheter passes through the ureteral orifice, advance the catheter to the desired location within the ureter.
5. Remove the stylet while maintaining the position of the catheter.

6. Under fluoroscopy, inject contrast media through the catheter to visualize the ureter and to determine required intervention.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

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© COOK 2019
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd.