EN 3	Perc NCircle, Perc NCompass, Perc NGage Instruction for Use
CS 3	Perc NCircle, Perc NCompass, Perc NGage Návod k použití
DA 4	Perc NCircle, Perc NCompass, Perc NGage Brugsanvisning
DE 5	Perc NCircle, Perc NCompass, Perc NGage Gebrauchsanweisung
EL 6	Perc NCircle, Perc NCompass, Perc NGage Οδηγίες χρήσης
ES 7	Perc NCircle, Perc NCompass, Perc NGage Instrucciones de uso
FR 8	Perc NCircle, Perc NCompass, Perc NGage Mode d'emploi
HU 8	Perc NCircle, Perc NCompass, Perc NGage Használati utasítás
IT 9	Perc NCircle, Perc NCompass, Perc NGage Istruzioni per l'uso
NL 10	Perc NCircle, Perc NCompass, Perc NGage Gebruiksaanwijzing
NO 11	Perc NCircle, Perc NCompass, Perc NGage Bruksanvisning
PL 12	Perc NCircle, Perc NCompass, Perc NGage Instrukcja użycia
PT 13	Perc NCircle, Perc NCompass, Perc NGage Instruções de utilização
SV 14	Perc NCircle, Perc NCompass, Perc NGage Bruksanvisning

PERC NCIRCLE, PERC NCOMPASS, PERC NGAGE

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Perc NCircle, Perc NCompass, and Perc NGage are rigid nitinol extractors. Nitinol basket configuration will vary, based upon extractor type.

INTENDED USE

The Perc NCircle, Perc NCompass, and Perc NGage are used for extraction of stones and foreign objects in the kidney and bladder.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

- This device is conductive. Avoid contact with any electrified instrument.
- Excessive force could damage device.

INSTRUCTIONS FOR USE

1. Using previously established kidney/bladder access, maintain direct vision with endoscope.
2. Insert the extractor into the endoscope working channel until the distal tip is visible.
3. While maintaining direct visualization of the stone or foreign object, open the extractor by squeezing the red handle, engaging the stone or foreign object in the extractor.
4. Slowly release the handle to close down on the stone or foreign object.
5. Slowly remove the endoscope and extractor simultaneously.
6. After the extractor is removed from the patient, squeeze the red handle to release the stone or foreign object.
7. Repeat the above steps as necessary for multiple extractions.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

PERC NCIRCLE, PERC NCOMPASS, PERC NGAGE

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© COOK 2013
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.