**Silhouette™ Transitionless
Peel-Away® Introducers**

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**IT
7**

**Silhouette™ overgangsløse
Peel-Away® introducers**

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**ZH
10**

T - P L V T W - R E V . O

SILHOUETTE™ TRANSITIONLESS PEEL-AWAY® INTRODUCERS

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

Silhouette Transitionless Peel-Away Introducers are constructed of radiopaque TFE material and feature a transitionless profile between the sheath and dilator.

INTENDED USE

Silhouette Transitionless Peel-Away Introducers are intended for percutaneous placement of external and internal drainage catheters constructed from softer materials.

The product is intended for use by physicians trained and experienced in diagnostic and interventional techniques. Standard techniques for placement of access sheaths, catheters, and wire guides should be employed.

CONTRAINDICATIONS

None known

WARNINGS

The product is not intended for intravascular use.

PRECAUTIONS

- When advancing introducer, use caution to avoid damage to adjacent tissues.
- Take care to maintain catheter position while peeling introducer away.
- **This introducer has a uniquely fabricated transition between dilator and sheath. DO NOT turn, pull back, or in any way manipulate dilator prior to introduction.**
- **Do not attempt to flush sheath prior to insertion.**

INSTRUCTIONS FOR USE

1. Introduce access needle and advance to desired location. Confirm position of needle tip using fluoroscopy.
2. Pass flexible tip of wire guide through needle and advance wire guide at least 10 cm beyond needle tip.
3. Remove needle, maintaining wire guide position.
4. If necessary, dilate tract using appropriately sized dilator(s).
5. **Pass Peel-Away Introducer over wire guide and advance into desired location.**
6. **Turn dilator one half turn.**
7. **Pull back dilator to remove from sheath.**
8. Leaving sheath portion of introducer in place, remove wire guide.
9. Insert catheter through sheath and advance into position.
10. Grasp knobs of sheath and pull outward and upward to peel sheath away from catheter. **NOTE:** Take care to maintain catheter position while peeling sheath away.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.