COOK

MEDICAL

EN

3

Polyethylene Catheters

Instructions for Use

FR-CA

5

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Mode d'emploi

T _ P O L Y _ R E V 1

POLYETHYLENE CATHETERS

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

Polyethylene Catheters are for use in angiographic procedures and available in a variety of lengths and tip configurations.

INTENDED USE

The Polyethylene Catheters are intended for use in angiographic procedures.

CONTRAINDICATIONS

3 French catheters should not be used with power injection.

WARNINGS

None known

PRECAUTIONS

- This catheter is intended for use in angiographic procedures by physicians trained and experienced in angiographic techniques. Standard techniques for placement of vascular sheaths, angiographic catheters and wire guides should be employed.
- Manipulation of the catheter requires fluoroscopic control.
- If resistance is encountered during manipulation, stop and determine the cause before proceeding any further.
- Do not attempt to heat or reshape the catheter curve; the catheter tip is made from a heat-sensitive material.

POWER INJECTION RECOMMENDATIONS (5 FRENCH–7 FRENCH CATHETERS ONLY)

- The labeled pressure is the maximum power injector machine settings.
- Labeled flow rate achieved with 12.7cP contrast and contrast temperature of 22°C ± 2°C.

French	Flow Rate (cc/sec)	Maximum Power Injector Setting (bar) / (psi)
P3.0	Hand injection only	
P5.0	10	43 / 630
P5.0B	11	40 / 585
P6.7	27	52 / 765
P7.1	35	55 / 810
P7.2	35	68 / 990

INSTRUCTIONS FOR USE

1. Inspect the packaging and the device to verify no damage. Do not use product if the catheter or packaging is damaged.
2. Flush the catheter prior to use.
3. Under fluoroscopic guidance, advance the catheter over an appropriately sized wire guide or through an appropriately sized sheath to the intended location. **NOTE:** If resistance is encountered during manipulation, stop and determine the cause before proceeding any further.
4. When using a contrast power injector, ensure all connections are secure. Please refer to the Power Injection Recommendations section for pressure and flow rate information. **NOTE:** 3 French catheters should not be used for power injection.
5. To remove the catheter upon completion of use, reinsert the wire guide. Slowly withdraw the catheter over the wire guide or through the sheath, using fluoroscopic guidance as needed.
6. Upon completion of the procedure, dispose of the device per institutional guidelines for biohazardous medical waste.

Catheters with pigtail shaped or similar configurations are supplied with Peel-Away straighteners to be used as follows.

Fig. 1

- a) Slide the Peel-Away straightener forward to open the catheter tip. (Fig. 1)

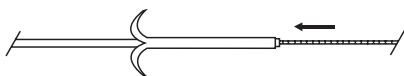
Fig. 2

b) With catheter tip straight, advance the catheter over the wire guide. (**Fig. 2**)

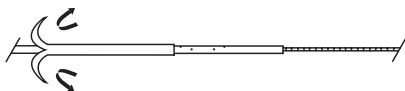
Fig. 3

c) After the initial advancement of the catheter over the wire guide, pull back and peel away the straightener for catheter advancement. (**Fig. 3**)

NOTE: The Peel-Away straightener must not be used as a vascular introducer sheath.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.