Pigtail Ureteral Catheter Set

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EN
3
CS
3
DA
4
DE
5
EL
5
ES
6
FR
7
HU
7
IT
8
NL
9
NO
10
PL
10
PT
11
SV
12
ZH
12

PIGTAIL URETERAL CATHETER SET

Used for drainage and irrigation. The pigtail tip configuration aids in retention of the distal tip in the renal pelvis. Intended for one-time use.

SET CONTAINS

- Radiopaque polyurethane pigtail catheter
- PTFE coated stainless steel wire guide

NOTE: If no wire guide is provided with this set, the following is recommended: Catheters accept .038" PTFE wire guide.

NOTE: Prior to use, immerse AQ® catheter in sterile water or isotonic saline to allow the hydrophilic surface to absorb water and become lubricious. This will ease placement under standard conditions.

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

There are no known contraindications.

SUGGESTED INSTRUCTIONS FOR USING PIGTAIL URETERAL CATHETER SET

1. Pass the flexible end of the wire guide into the renal pelvis. **CAUTION: Care must be taken not to pass the wire guide too far into the renal pelvis. The flexible tip of the wire guide is 3 cm long.**
2. While holding the wire guide in a set position, pass the ureteral catheter, pigtail first, over the wire guide through the ureter and into the renal pelvis.
3. Verify that the pigtail is in the correct position by standard radiographic procedure or fluoroscopy. The pigtail should be in the center of the renal pelvis.
4. Remove the wire guide and then the cystoscope while maintaining the catheter position.
5. The supplied connector can be attached to the catheter for irrigation or connection of a drainage system.

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© COOK 2019
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd