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T _ R B E _ R E V O

RUTNER BALLOON DILATION STONE EXTRACTOR

Used for ureteral dilation and stone manipulation in the ureter. Intended for one-time use.

DESCRIPTION

- Radiopaque balloon catheter 65 cm long
- Stainless steel basket to fit balloon catheter
- Endoscopic cap
- Syringe

CAUTION: This device must not come in contact with any electrified instrument.

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CONTRAINDICATIONS

There are no known contraindications.

SUGGESTED INSTRUCTIONS FOR USING RUTNER BALLOON DILATION STONE EXTRACTOR

CAUTION: Balloon dilation catheters are intended for use by physicians trained and experienced in techniques for balloon-catheter dilation.

WARNING: Do not preinflate balloon.

1. Remove the protective sleeve from the balloon and discard. Lock the syringe onto the side-arm stopcock of the balloon catheter. While folding the balloon counterclockwise, aspirate until all air is removed. Close the stopcock.
2. Retract the basket into the balloon catheter; lubricate the deflated balloon with a water soluble lubricant.
3. Draw a small amount of dilute contrast medium (30% solution) into the syringe. Eliminate air from the syringe.
4. Attach the endoscopic cap to the accessory channel of the cystoscope.
5. Cystoscopically pass the extractor, under X-ray or fluoroscopic control, until the balloon is within the distal ureter and projects only a short distance into the bladder through the ureteral orifice. **NOTE: A 23.0 French or larger scope is recommended.**

WARNING: Always inflate the balloon with a sterile liquid. Never inflate with air, carbon dioxide or any other gas.

6. Lock the syringe onto the stopcock. Open the stopcock; using light to moderate thumb pressure, inject contrast medium.

NOTE: To ensure proper regulation of balloon pressure, use of a balloon inflation device and pressure gauge is recommended.

CAUTION: To prevent the balloon from rupturing, avoid excessive pressure. Maximum inflation pressure for the 5 mm balloon is 40 psi; maximum inflation pressure for the 6 mm balloon is 60 psi. Do not overinflate the balloon.

7. Close the stopcock for at least 30 seconds to keep the balloon inflated and to allow for adequate ureteral dilation. **NOTE: Time, rather than excessive pressure, is the key factor.**
8. When the stone is higher than the upper end of the balloon, deflate the balloon using syringe aspiration. Reposition the balloon at a higher level and repeat the process. After the ureter section from the stone to the ureteral orifice has been dilated for at least 30 seconds, open the basket above the stone.
9. Deflate the balloon and engage the stone. Carefully withdraw the stone into the bladder. **NOTE: When indicated, a limited ureteral meatotomy is performed by cutting the roof of the ureteral meatus between the visualized basket wires.**

REFERENCE

A. B. Rutner: "Balloon-Dilation of the Lower Ureter to Facilitate Cystoscopic Extraction of Large Ureteral Calculi," *Urology*, 21 (1983), 226-231.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.