EN
3

Transjugular Renal Catheter Needle

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TRANSJUGULAR RENAL CATHETER NEEDLE

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Transjugular Renal Catheter Needle consists of a transjugular biopsy needle and a catheter. Refer to product label for further information.

INTENDED USE

The Transjugular Renal Catheter Needle is designed for transjugular approach to renal biopsy in patients with conditions usually considered as contraindications for percutaneous biopsy (e.g., bleeding disorder and uncontrolled hypertension).

CONTRAINDICATIONS

None known

WARNINGS

Repeated biopsy in the same venous branch should be limited to three attempts in order to minimize the risk of excessive bleeding.

PRECAUTIONS

- The procedure should be performed under fluoroscopic guidance.
- Do not insert the needle into the catheter without using a wire guide. Damage to the catheter may result.

INSTRUCTIONS FOR USE

1. Inject local anesthesia and make a small skin incision at the planned jugular vein puncture site.
2. Attach a saline-filled syringe to a needle and puncture the jugular vein. After puncture, carefully withdraw the needle, while aspirating, until venous blood is seen.
3. Remove the syringe and introduce a wire guide through the needle; advance the wire guide until it reaches the inferior vena cava.
4. Remove the needle, leaving the wire guide in place.
5. Advance a 9.0 French introducer set over the wire guide, then remove the introducer, leaving the sheath in place.
6. Introduce the catheter over the wire guide into the sheath; advance the catheter into the inferior vena cava.
7. After advancing the wire guide, advance the catheter tip directly into the renal vein. **NOTE:** The oblique and sometimes nearly vertical course of the right renal vein is often favorable to insertion of the biopsy needle.
8. Remove the wire guide and inject contrast medium to confirm position.
9. Introduce the wire guide into the hub of the transjugular needle and advance until the wire protrudes from the needle tip.
10. Insert the needle/wire assembly into the catheter; advance the assembly until it reaches the catheter tip. **NOTE:** Do not insert the needle into the catheter without using the wire guide. Damage to the catheter may result.
11. With the needle tip still inside the catheter, and the catheter still positioned in the selected renal vein, remove the wire guide.
12. Attach a saline-filled syringe to the transjugular biopsy needle. While the patient holds his/her breath in aspiration, advance the catheter/needle assembly to a wedged position.
13. Holding the catheter steady, advance the needle with a thrust beyond the tip of the catheter into the renal tissue.
14. Immediately withdraw the needle while applying suction to the syringe. Suction should be discontinued just before the needle tip is pulled back into the renal vein to ensure that the tissue sample remains in the needle.
15. Withdraw the needle and gently flush out the specimen with saline. Repeat steps 9-15 as needed to obtain samples. **CAUTION: Repeated biopsy in the same venous branch should be limited to three attempts in order to minimize the risk of excessive bleeding.**
16. When sufficient samples have been obtained, remove the catheter and sheath.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd