EN

2

Roadrunner® Biliary Wire Guide

Instructions for Use

ES

3

Guía biliar Roadrunner®

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FR

4

Guide biliaire Roadrunner®

Mode d'emploi

T _ R R B W G _ R E V O

ROADRUNNER® BILIARY WIRE GUIDE

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

The Roadrunner® Biliary Wire Guide is coated with AQ (a biocompatible hydrophilic coating), which when activated becomes lubricious. The wire guide is made of a nitinol core with a distal platinum inner spring coil tip, which is completely covered by a bismuth-impregnated polymer jacket.

INTENDED USE

The Roadrunner Biliary Wire Guide assists in cannulation for laparoscopic and general surgical procedures for the dilation of the cystic duct to facilitate common bile duct exploration.

CONTRAINDICATIONS

- Peritonitis
- Severe cholangitis
- Severe acute pancreatitis
- Known or suspected perforation of the cystic and/or bile ducts

WARNINGS

- Altering the tip's configuration or curve manually may damage the wire guide.
- Excessive tightening of a torque device may abrade the coating on the wire guide.
- This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.
- If package is opened or damaged when received, do not use. Visually inspect with particular attention to kinks, bends and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook for return authorization.
- Carefully read all instructions prior to use. Failure to observe all warnings and precautions may result in complications.

PRECAUTIONS

- This product is intended for use by physicians trained and experienced in diagnostic and interventional techniques. Standard techniques for biliary access and procedures should be employed.
- Always keep the surface of the wire guide wetted for optimal results.
- Roadrunner wire guides are very slippery when wet. Always maintain control of the wire guide when manipulating it through the catheter.

INSTRUCTIONS FOR USE

Activating the AQ® Hydrophilic Coating

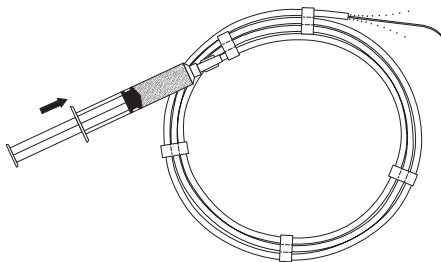
Fig. 1

1. Attach the syringe with normal saline solution or sterile water to the Luer lock fitting of the wire guide holder. (**Fig. 1**)
2. Inject enough solution or sterile water to wet the wire guide surface entirely. This will activate the AQ coating, making the wire guide surface very lubricous.
3. Attach the torque device to the wire guide.
4. Standard wire guide use may now be initiated. An initial flush of normal saline or sterile water through the catheter lumen will enhance initial movement of the wire guide through the catheter.

NOTE: If the surface of the wire guide becomes dry after removal from the holder or catheter, wetting with additional normal saline or sterile water will renew the hydrophilic effect.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.