

COOK

MEDICAL

San Antonio Stopcock

Instructions for Use



T _ S A S _ R E V O

SAN ANTONIO STOPCOCK

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

The San Antonio Stopcock is an endoscopic adapter designed to facilitate irrigation during a procedure. The adapter can be connected to any standard female Luer lock fitting.

INTENDED USE

This device is intended to regulate flow through the working channel of a ureteroscope, cystoscope, or other endoscope.

CONTRAINDICATIONS

None known

WARNINGS

For single use only. Do not reuse, reprocess, or resterilize.

PRECAUTIONS

This device is intended for use by physicians trained and experienced in urological procedures. Standard techniques should be employed.

INSTRUCTIONS FOR USE

1. Introduce the adapter onto the working channel of an endoscope via a standard female Luer lock fitting.
2. Introduce irrigation tubing over the barbed fitting to allow for irrigation during the intended procedure.
3. After completion of the procedure and removal of the irrigation tubing, carefully remove the device from the endoscope.

Upon completion of the procedure, dispose of the device per institutional guidelines for biohazardous medical waste.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

A symbol glossary can be found at
<https://cookmedical.com/symbol-glossary>



MANUFACTURER

COOK INCORPORATED
750 Daniels Way
Bloomington, IN 47404 U.S.A.

www.cookmedical.com
© COOK 2018

2018-05
T_SAS_REV0