MANUFACTURER
COOK INCORPORATED
750 Daniels Way
Bloomington, Indiana 47404, U.S.A.

www.cookmedical.com
© COPYRIGHT COOK 2013

EC REPRESENTATIVE
COOK IRELAND LTD.
O'Halloran Road
National Technology Park
Limerick, Ireland

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DEVICE DESCRIPTION 2

Components and Accessories 2

INTENDED USE 3**WARNINGS 3****PRECAUTIONS 3****POTENTIAL ADVERSE EFFECTS 3****INSTRUCTIONS FOR USE 3**

Preparation For Use 3

Preparation for Cleaning, Disinfection and Sterilization 3

Cleaning 3

Disinfection 4

Sterilization 4

PROCEDURE 4

General 4

Pre-use Preparation 4

Intracorporeal Lithotripsy Procedure 4

MAINTENANCE 5**EFFECTS ON ENVIRONMENT 5****RETURN TO MANUFACTURER 5****WARRANTY STATEMENT 5****TROUBLESHOOTING 6****TECHNICAL DATA 7**

StoneBreaker and Probes 7

StoneBreaker CO₂ Cartridge 7

StoneBreaker™ Pneumatic Lithotripter

Read all instructions carefully prior to use. Failure to properly follow instructions, warnings, and precautions can affect device performance and patient safety.

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The StoneBreaker is a portable, non-electrical, compact intracorporeal lithotripter, intended to fragment stones in the urinary tract (i.e. kidney, ureter and bladder). It weighs around 500g with a probe and it requires no extraneous electrical or pneumatic connections. It is powered by a detachable cartridge of high-pressure carbon dioxide gas, with a maximum pre-adjusted operating pressure of 31 bar. The mechanical shock generated by the device is transferred as kinetic energy through the length of the probe to the tip, which is in direct contact with the stone to be fragmented. The advantage of delivering high-pressure at contact is to allow expeditious fragmentation, thereby reducing operative time. There is very little movement of the probe itself, allowing for safe stone fragmentation.

A StoneBreaker carbon dioxide (CO₂) gas cartridge typically provides the necessary energy for one surgical procedure. Once the cartridge is perforated by the built-in screw system, the compressed gas passes through a pre-adjusted pressure regulator in the device and a mechanical shockwave required for fragmentation of urinary stone is generated when the trigger is depressed. Expended gas will leave the device through a specially designed exhaust port and the attached exhaust line. The trigger snaps back to its starting position when the gas is expended through the exhaust port, and the device is then ready for further use.

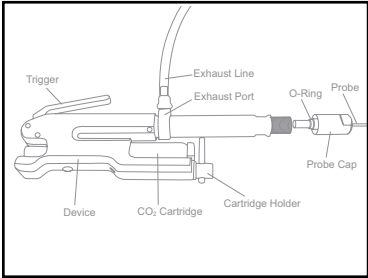
Figure 1

One full StoneBreaker carbon dioxide gas cartridge should allow delivery of at least 80 shocks. The cartridge must be removed from the device after the completion of the surgical operation to allow complete release of gas from the cartridge. The StoneBreaker is recommended for use with endoscopes equipped with a straight working channel.

Components and Accessories

The StoneBreaker device is made up of the following main components. (See Figure 1)

- Trigger – Activates the mechanical shockwave required for fragmentation of the calculus.
- Probe Cap – Holds the probe in place.
- Cartridge Holder – Locks and holds the carbon dioxide gas cartridge in place.
- Carbon Dioxide Gas Cartridge – Energy source. **NOTE:** The StoneBreaker carbon dioxide gas cartridges, which are sterilized by Gamma irradiation and are supplied separately from the device, are surface-sterile only; the gas within the cartridge is not sterile.

CAUTION: The gas cartridge must be removed from the device after the completion of each surgical procedure.

- Exhaust Port – Exhaust of expended carbon dioxide gas from the device.
 - Exhaust Line – Attached to the exhaust port to lead the expended gas away from the field of operation.
 - Probe – The StoneBreaker Probes are supplied in separate stand-alone packaging. The probes are single-use and are sterilized by ethylene oxide.
- Table 1 indicates the current StoneBreaker Probes available for use with the device. Please refer to the product catalog for the most up-to-date available product range.

Table 1: Specification of StoneBreaker Probes

Description	Diameter	Total Length
Probe 1.0 mm	1.0 mm (3 Fr)	500 mm (19.7")
Probe 1.0 mm	1.0 mm (3 Fr)	605 mm (23.82")
Probe 1.6 mm	1.6 mm (4.8 Fr)	500 mm (19.7")
Probe 1.6 mm	1.6 mm (4.8 Fr)	605 mm (23.82")
Probe 2.0 mm	2.0 mm (6 Fr)	425 mm (16.73")

The following accessories are included with the StoneBreaker:

- Exhaust Cap – Closes the exhaust port to prevent moisture from entering the device during cleaning, disinfection and sterilization.
- Sterilization Cap – Replaces the carbon dioxide gas cartridge to prevent moisture from entering the device during cleaning, disinfection and sterilization.

CAUTION: Use only StoneBreaker CO₂ cartridges and probes; the use of the wrong gas cartridges and/or probes can affect the device performance and patient safety.

INTENDED USE

The StoneBreaker is intended for use via a rigid or semi-rigid endoscope with a straight working channel, for the fragmentation of urinary (i.e. kidney, ureter and bladder) tract stones.

WARNINGS

- Ensure that the probe tip does not exceed the endoscope channel at the time of introducing the endoscope into the urinary tract.
- The StoneBreaker is delivered non-sterile, and it must be cleaned, disinfected and sterilized before initial use and before each subsequent use.
- For the StoneBreaker Probe Single Use (sterile) : Do not use if the unit pouch has been previously opened or damaged.
- Do not use the single-use probe more than once. Re-usage of single-use probe increases the risk of patient injury, damage to device or endoscope or may lead to device malfunction.
- For the StoneBreaker carbon dioxide gas cartridge (sterile), do not use if the unit pouch has been previously opened or damaged.
- Do not use the StoneBreaker or any of the accessories if they are damaged in any way.
- Do not use the StoneBreaker Probe if the o-ring is missing or if it is damaged in any way.
- Do not autoclave the carbon dioxide gas cartridge with the StoneBreaker, or attempt to resterilize the gas cartridge in any way.
- Never attempt to refill the empty carbon dioxide gas cartridge.
- Never attempt to transfer gas from one carbon dioxide gas cartridge to another.
- Never attempt to disassemble the device.

PRECAUTIONS

- The StoneBreaker must be used only by specialists trained in endourology and should be used only for the purposes intended. In case of malfunction or failure of the device, it is recommended that the surgeon have another unit (or other intracorporeal lithotripsy device) on standby in order to complete the treatment scheduled at the time.
- It is recommended that urologists who have not used an intracorporeal lithotripter before to receive training from a Cook sales representative in order to learn how to manipulate the device.
- It is recommended that extra probes be available in case of malfunction.
- Ensure the probe size chosen is compatible with the endoscope's working channel and appropriate for the clinical condition that is being treated.

POTENTIAL ADVERSE EFFECTS

Due to the movement of the endoscope and high force generated by the StoneBreaker device, there is the potential of tissue injuries, should the probe tip be in direct contact with the mucosal membrane, causing perforation of tissue, pain, bleeding, renal damage and/or infection.

Review of published literature showed that adverse events that may potentially occur with the use of intracorporeal lithotripter include perforation of the bladder or ureter, bleeding in urinary tract or blood in the urine, pain, renal damage and infection.

Breakage of the probe during use, though unlikely, may potentially occur. The practitioner must then determine the most effective way of intervention, following all relevant procedures for such intervention.

INSTRUCTIONS FOR USE

Preparation for Use

Before each use, it is necessary to clean, disinfect and sterilize the StoneBreaker device and exhaust line. Proper cleaning, disinfection and sterilization are essential to ensure continued safe use of the device and accessories.

NOTE: The StoneBreaker Probe Single Use (sterile) and StoneBreaker carbon dioxide gas cartridge (sterile) are individually packed, and ready to use. The following procedure on cleaning, disinfection and sterilization is not applicable to these sterile, "ready-to-use" products.

Preparation for Cleaning, Disinfection and Sterilization

All cleaning, disinfection and sterilization methods listed have been validated. Ensure the probe is not attached to the device. If the probe is still present, unscrew the probe cap to remove the probe. Re-attach the probe cap back onto the device. Refer to **Figure 2**. Ensure the sterilization cap and exhaust cap are screwed tightly in place at the cartridge holder and exhaust port respectively. This is to prevent moisture from entering the device during cleaning, disinfection and sterilization.

NOTE: It is possible for moisture to adversely affect the effective performance of the device. If liquid has penetrated the device, return it to the manufacturer for servicing.

The hospital's own protocol for cleaning, disinfection and sterilization of surgical instruments should be followed whenever possible. The following is the recommended procedure for cleaning, disinfecting and sterilizing the StoneBreaker device and exhaust line.

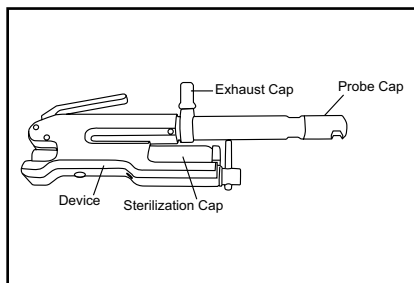
Figure 2

Cleaning

Prepare an enzymatic cleaning agent per the manufacturer's instructions. Submerge and soak the StoneBreaker device and its accessories into an enzymatic cleaning agent for 2-5 minutes. After soaking the device and its accessories, thoroughly wash the device and its accessories by gently cleaning with a soft bristle brush in the enzymatic cleaning solution until all visible debris is removed. Thoroughly rinse the device with a minimum of 3 liters of water to remove cleaning residue. Carefully inspect to ensure that all visible foreign matter has been removed. A specific cleaning agent found to be compatible with the StoneBreaker and its accessories is Endozime® (Ruhof, Valley Stream, NY).

CAUTION: Automated cleaning methods are not compatible with the StoneBreaker and its accessories.

Disinfection

CAUTION: Ensure the device is watertight by tightening the sterilization cap and exhaust cap at their respective locations.

One disinfecting solution that is found to be compatible with the StoneBreaker device and accessories is Cidex® OPA (Johnson & Johnson, UK). Submerge the device and accessories into the Cidex® OPA at 20-25°C for at least 20 minutes. Thoroughly rinse the disinfected device with one liter of sterile water three times and dry it.

CAUTION: Automated disinfection methods are not compatible with the StoneBreaker and its accessories.

Sterilization

Steam autoclaving is the only recommended method for sterilizing the StoneBreaker device and exhaust line. The device and its accessories should be placed in a suitable sterilization bag or tray prior to placing them in a steam autoclave. The two steam sterilization cycles that are suitable for the StoneBreaker device and exhaust line are:

- 134°C for 10 minutes in a pre-vacuum autoclave with 30-minute dry time
- 121°C for 30 minutes in a pre-vacuum autoclave with 30-minute dry time

Remove the probe cap and ensure that the sterilization cap and the exhaust cap are securely in place prior to sterilizing the device. Place the probe cap and the exhaust line in the tray with the StoneBreaker for sterilization.

WARNING: Do not sterilize or resterilize the CO₂ cartridge in any way; cartridge can explode.

CAUTION: After autoclave sterilization, do not utilize the StoneBreaker in a warm condition, as this may cause damage to the components within. The following minimum cooling period is required: EITHER ambient air cooling period - 60 minutes minimum OR sterile water cooling period - 5 minutes minimum.

PROCEDURE

General

The StoneBreaker requires effective endoscope manipulation in order to ensure proper direct contact of the probe with the calculus, with the probe tip protruding from the endoscope working channel by 10 mm to 20 mm maximum. The choice of the endoscope depends on the position of the stone (kidney, ureter or bladder). The StoneBreaker is recommended for use with a rigid or semi-rigid endoscope equipped with a straight working channel, and to use the largest probe possible to allow transmission of maximum energy. **Table 2** shows the recommended diameter for the endoscope working channel with the respective StoneBreaker probes.

Table 2: Recommended Endoscope Working Channel Diameter

Probe Diameter	Diameter of Endoscope Working Channel
2.0 mm (6 Fr)	>2.33 mm (7 Fr)
1.6 mm (4.8 Fr)	>1.83 mm to 2 mm (5.5 to 6 Fr)
1.0 mm (3 Fr)	>1.17 mm to 1.8 mm (3.5 to 5.4 Fr)

NOTE: It is recommended that for better and simplified alignment of the working channel and the StoneBreaker Probe, endoscope adaptors that are available from the endoscope manufacturers should be used.

NOTE: Each carbon dioxide gas cartridge should provide sufficient energy for one typical procedure.

Pre-use Preparation

Prior to the use of the StoneBreaker for stone fragmentation, it is recommended that the following checks be conducted:

1. Ensure that the StoneBreaker device and its accessories are not damaged in any way.
2. For the StoneBreaker Probe Single Use (sterile) and the StoneBreaker carbon dioxide gas cartridge (sterile), check to ensure the unit pouch is not damaged or previously opened. Do not use if the unit pouch is damaged or previously opened.
3. Remove both the sterilization cap and exhaust cap from the StoneBreaker device.
4. Attach the exhaust line onto the exhaust port.
5. Ensure the o-ring is in the correct position on the probe.
6. Remove the probe cap from the device and insert the probe through the opening in the probe cap. Replace the probe cap to the device and ensure that the probe does not move too freely.
7. Insert a new StoneBreaker carbon dioxide gas cartridge into the cartridge holder and tighten to hold the cartridge in place but WITHOUT perforating it.

CAUTION: Do not perforate the cartridge until immediately before advancing the probe through the endoscope to treat the stone. Leaving a perforated carbon dioxide gas cartridge in the device for too long may adversely affect the performance of the device.

Intracorporeal Lithotripsy Procedure

Before the operation, tighten the cartridge holder until the cartridge is perforated. The trigger should now be under pressure. Turn the lever of the cartridge holder a quarter round more to secure the cartridge to the device.

CAUTION: Ensure that the device is working correctly by depressing the trigger two to three times, and ensure the trigger returns to the original position every time the trigger is released. Position the exhaust line away from the head of the operator and ensure it is properly attached to the exhaust port.

CAUTION: Exhaust line disconnection from the exhaust port during use may indicate malfunction of the pressure regulator.

1. Position the probe, through the endoscope working channel, to be in direct contact with the stone.

CAUTION: Ensure proper alignment of the probe and endoscope, by ensuring the probe is straight and extends beyond the end of the endoscope. Any misalignment may damage both the probe and endoscope. It is recommended that the practitioner ensure that the tip of the probe remains in sight throughout the procedure.

CAUTION: Ensure the probe is extended beyond the tip of the endoscope by 10 mm to 20 mm to avoid damage to the endoscope.

2. When ready, hold the device firmly in place and depress the trigger of the StoneBreaker. This will generate the mechanical shock required to fragment the targeted stone.

WARNING: Ensure minimal movement of the device and probe. Any significant movement of the device and probe during stone fragmentation may result in direct contact of the activated probe with tissue resulting in tissue damage and / or bleeding.

3. Release the trigger and allow it to return to its original position after each depression.

CAUTION: If the trigger cannot be depressed, it may indicate malfunction of the pressure regulator.

4. For subsequent shocks, reposition the device and hold it firmly before depressing the trigger as required.

NOTE: One new full StoneBreaker carbon dioxide cartridge should allow delivery of at least 80 shocks.

5. Once the procedure is complete, remove the carbon dioxide cartridge from the cartridge holder by loosening the lever slowly. This will allow any remaining gas in the cartridge to escape, thereby emptying the cartridge.

CAUTION: Do not sterilize the spent cartridge with the device.

MAINTENANCE

Though the StoneBreaker is designed and manufactured to require minimum maintenance, proper care and maintenance are required to ensure safe and effective use of the device. The manufacturer recommends that the device and its accessories be carefully checked upon receipt and prior to each use to prevent injury to the patient or user. Follow the recommendations listed in this IFU for proper routine maintenance and care of the device.

Defective probes (such as those that are kinked or bent, or have dent marks along the length of the probe) should be discarded. Other components that may be damaged and lose their efficiency in maintaining water-tightness of the device are the sterilization cap and exhaust cap. Check the integrity of these components before using them on the device.

EFFECTS ON ENVIRONMENT

This device does not contain any harmful components; follow local statutory requirements / instructions for proper disposal of the non-functioning devices, probes and other accessories. Never dump the empty carbon dioxide cartridges at sea. The cartridge is made of zinc-plated and chrome-stabilized steel, and should be properly disposed of in accordance with local statutory requirements and instructions.

CAUTION: Never dispose of an unused carbon dioxide cartridge. Ensure that the cartridge is empty before disposal.

RETURN TO MANUFACTURER

Should there be a need to return the StoneBreaker to the manufacturer for investigation or repair, please ensure the product is properly cleaned, disinfected and sterilized in accordance to the recommendations listed in this IFU before shipment.

Please return the device in its sterilization bag (as evidence that the device is sterile), whenever possible. The manufacturer may refuse receipt of the device and / or accessories if it is found that the products are not properly disinfected or sterilized or there is no evidence that these procedures have been carried out.

WARRANTY STATEMENT

Cook Incorporated ("Company") warrants to the purchaser of this product that at the time of manufacture, the product was manufactured and tested in accordance with Good Manufacturing Practices specified by the United States Food and Drug Administration and was true to label. In the unlikely event of product failure due to defects in material or workmanship within the warranty period, products will be repaired or replaced, at Cook's option, free of charge.

Because the Company has no control over the conditions under which this product is used, diagnosis of the patient, anatomical or biological differences among patients, method of use or administration, and handling of the product after it leaves our possession, we do not warrant a good effect or against an ill effect following the product's use. The foregoing warranty is exclusive and in lieu of all other warranties whether written, oral, or implied. There are no warranties of merchantability or fitness for a particular purpose. No representative of the Company may change any of the foregoing, and the purchaser accepts the product subject to all terms hereof. The Company reserves the right to change or discontinue this product without notice.

Limits and Exclusions

The warranty does not cover consumables such as probes, CO₂ cartridges, exhaust line, exhaust cap, sterilization cap and probe cap.

The warranty does not apply in the event of misuse, negligence, or accidental damage of product. The StoneBreaker is designed only for use with Cook brand products that are specified for use with the StoneBreaker. Failure to use Cook brand products may constitute misuse or negligence and compromise the warranty. Additionally, the following events are specifically excluded from warranty coverage: steam sterilizing or submerging the device in any liquids without the proper sterilization caps in place.

The warranty will be void if an unauthorized service person has attempted to service the product or unauthorized modifications have been made to the product.

Warranty services will be provided only to the original purchaser at the location where the product was originally installed. The warranty may not be transferred without the prior written consent of the Company.

Warranty Period

This limited warranty for the product begins on the date the product ships from the Company's facilities and ends on the same day 12 months later. For replacement products provided during a warranty period, the remaining portion of the original warranty period shall apply to the replacement product.

Warranty Procedure

Many product issues can be resolved remotely. If you purchased your product from a distributor, please contact your distributor for technical assistance, repair and replacement assistance.

If you purchased the product from Cook, before returning a product to Cook for service or repair, please contact Cook to speak with a Cook service technician. If an issue cannot be resolved remotely, Cook may, at its discretion, repair or replace the defective product or component at customer's location or at a Cook-designated repair facility. If the repairs will be made at a Cook-designated repair facility, purchaser is responsible for the cost of shipping of the product to Cook's designated repair facilities, and Cook will pay return shipping back to the purchaser.

Purchaser must ensure the product is properly cleaned, disinfected and sterilized in accordance to the recommendations listed in the owner's manual before shipment. Please return device in its sterilization bag (as evidence that the device is sterile), whenever possible. The manufacturer may refuse receipt of the device and/or accessories if it is found that the products are not properly disinfected or sterilized or there is no evidence that it has been carried out. Customer is responsible for appropriate packaging, insurance, and risk of damage during transit to Cook's designated repair facility. Warranty does not guarantee repair response times.

TROUBLESHOOTING

Description	Possible Causes	Action
No shock generated	CO ₂ cartridge not perforated	Tighten the cartridge holder
	CO ₂ cartridge is empty	Replace the cartridge
	Device malfunction	Return to the manufacturer
	Probe Cap not properly in place	Tighten the probe cap
	Trigger not fully depressed	Depress the trigger fully
Shock generated is weak or without force	CO ₂ cartridge is nearly empty	Replace the cartridge
	CO ₂ cartridge not perforated properly	Tighten the cartridge holder fully
	Device malfunction	Return to the manufacturer
Gas leakage	Cartridge not properly secured in the cartridge holder	Tighten the cartridge holder fully
	Device malfunction	Return to the manufacturer
Shock generated has a very long excursion	Probe O-Ring is missing or damaged	Replace the probe
No shock generated at the tip of the probe, when tested against a sterile gloved finger	Probe O-Ring is missing or damaged	Replace the probe
Bad quality of the shock at the tip of the probe	Probe O-Ring is missing or damaged	Replace the probe
Trigger activation too stiff or difficult to depress	Pressure regulator may be defective	Return to the manufacturer
Trigger returns to original position too slowly	CO ₂ cartridge is empty or nearly empty	Replace with a new cartridge
	Device malfunction	Return to the manufacturer
Exhaust line constantly gets disconnected from exhaust port when trigger is depressed	Exhaust line not properly attached to exhaust port	Tighten the exhaust line to exhaust port
	Pressure regulator may be defective	Return to the manufacturer
Black particles found inside the exhaust line after use	Foreign particles present within device, but the device is functioning well	Change the exhaust line

TECHNICAL DATA

StoneBreaker and Probes

Operating mode	Intermittent, manually controlled by user			
Dimensions	Device	246 mm x 80 mm		
	Probe	Description	Diameter	Length
		Probe 1.0 mm	1.0 mm (3 Fr)	500 mm (19.7")
		Probe 1.0 mm	1.0 mm (3 Fr)	605 mm (23.82")
		Probe 1.6 mm	1.6 mm (4.8 Fr)	500 mm (19.7")
		Probe 1.6 mm	1.6 mm (4.8 Fr)	605 mm (23.82")
		Probe 2.0 mm	2.0 mm (6 Fr)	425 mm (16.73")
Weight	500g			
Conditions for use	Temperature range	+10°C to +40°C		
	Relative humidity range	30% to 70% RH		
Transportation conditions	Temperature range	-10°C to +45°C		
	Relative humidity range	10% to 95% RH		

StoneBreaker CO₂ Cartridge

Content	Pressurized carbon dioxide gas (99.5% Purity)	
	UN code	1013
Dimensions	Ø19 x 83 mm	
Weight	11.5 g	
Conditions for Use	Not higher than +45°C	
Transportation conditions	UN Code UN Class Classification code	UN 2037 2.2 5A
	According to ADR chapter 3.2 schedule A column 6, the special provisions 191 and 303 apply: (transported product is not subject to the ADR, IMDG or RID regulations)	
Storage conditions	Temperature range	-30°C to +45°C
	Store in a well-ventilated area and in a location free from fire risk and away from sources of heat and ignition	

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