Stanford Biopsy Access Set

Instructions for Use

EN
3

Stanford biopsiadgangssæt

Brugsanvisning

DA
3

Stanford Biopsiezugangs-Set

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DE
4

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T _ S B S _ R E V . 0

STANFORD BIOPSY ACCESS SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Stanford Biopsy Access Set consists of an inner pigtail catheter with sideports and an appropriately sized, curved outer sheath.

INTENDED USE

The Stanford Biopsy Access Set is intended for introduction of a biptome for performance of a myocardial biopsy.

The product is intended for use by physicians trained and experienced in diagnostic and interventional techniques. Standard techniques for placement of vascular access sheaths, angiographic catheters and wire guides should be employed.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

- Manipulation of product requires fluoroscopic control.
- Do not attempt to heat or reshape the catheter curves; the catheter tips are manufactured from a heat-sensitive material.
- Due to thinwall construction, extreme care must be exercised during manipulation and withdrawal to avoid pulling catheter apart. Catheter insertion through a synthetic vascular graft should be avoided whenever possible.

INSTRUCTIONS FOR USE

1. Using fluoroscopic control, introduce the catheter set through the aortic valve to the left ventricle.
2. Advance the sheath into proper position. If necessary, correct the position after performing hand injection of contrast medium.
3. When correct position has been obtained, withdraw the inner pigtail catheter and insert a biptome through the outer sheath.
4. Perform biopsy in standard fashion.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.