EN
3

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VISTA TERMINAL
DO CATETER PRESO
COM ADESIVO

TEJPAD KATETER I
GENOMSKÄRNING

STAMEY PERCUTANEOUS SUPRAPUBIC CATHETER EXCHANGE SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

- Malecot catheter
- Two flexible stylets
- .038 inch (0.97 mm) diameter stainless steel wire guide
- Connecting tube

NOTE: Set components may vary.

INTENDED USE

Used to replace an existing 12.0 or 14.0 French Stamey-Malecot suprapubic catheter.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

- The indwelling time of this device should not exceed four weeks. Evaluation and replacement beyond four weeks must be considered.
- The potential effects of phthalates on pregnant/nursing women or children have not been fully characterized and there may be concern for reproductive and developmental effects.

INSTRUCTIONS FOR USE

1. Place the appropriate flexible stylet inside the situ Malecot catheter and secure its position with the Luer lock to straighten the Malecot wings.

NOTE: A flexible stylet is not used to remove an existing 10.0 French Stamey-Malecot catheter. The appropriate flexible stylet is used to straighten a 12.0 or 14.0 French catheter prior to wire guide insertion and catheter removal.

2. Introduce approximately half of the wire guide into the catheter/stylet.
3. While maintaining the wire guide in position, withdraw the Malecot catheter/stylet.
4. Place the second stylet inside the replacement Malecot catheter and secure its position with the Luer lock to straighten the Malecot wings.
5. Pass the Malecot end of the catheter over the external end of the wire guide; gradually advance the catheter into the bladder to the midpoint of the catheter shaft.
6. After releasing the Luer lock in order to open the Malecot wings, carefully remove the blunt stylet. Carefully remove the wire guide.
7. Use the connecting tube to connect the Malecot catheter to a drainage bag.
8. Allow the bladder to empty.
9. Withdraw the Malecot catheter until the Malecot wings meet the resistance of the bladder dome. Advance the catheter approximately 2 cm back into the bladder to allow for movement.
10. Apply tincture of benzoin to the Malecot catheter shaft and the abdominal skin.
11. With the catheter held up in a vertical position, begin at the skin level and place an 8 inch long strip of 2 inch wide tape vertically against the catheter tube.
12. Wrap the tape completely around the catheter tube, forming a 1 cm adhesive land to allow for catheter movement; lay the tape flat on the abdomen.

13. Apply a second piece of tape in the same fashion just above the first.
14. Using a 3 inch long piece of tape, secure the connecting tube to the abdomen in a gentle curve (**Fig. 1**).

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

REFERENCE

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

T. A. Stamey: "Suprapubic Cystostomy," Monographs in Urology, 1 (1980), editorial page.

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© COOK 2012
Bloomington, IN 47404 U.S.A.
COOK IRELAND LTD.