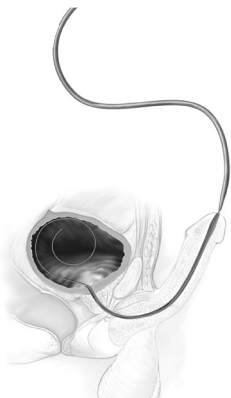
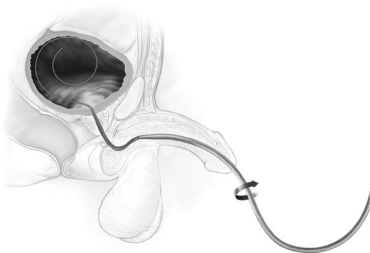
- EN**
3 **S~Curve Urethral Dilator Set**
Instructions for Use
- CS**
4 **Souprava ureterálních dilatátorů ve tvaru S**
Návod k použití
- DA**
5 **Dilatatorsæt til urethra med S-formet kurve**
Brugsanvisning
- DE**
6 **Urethra-Dilatatoren-Set mit S-Biegung**
Gebrauchsanweisung
- EL**
7 **Σετ διαστολέα ουρήθρας με καμπύλη σχήματος S**
Οδηγίες χρήσης
- ES**
8 **Equipo de dilatadores uretrales en S**
Instrucciones de uso
- FR**
9 **Set de dilatateur urétral à courbe en S**
Mode d'emploi
- HU**
10 **S alakú húgycsőtágító készlet**
Használati utasítás
- IT**
11 **Set di dilatatori uretrali curvati a S**
Istruzioni per l'uso
- NL**
12 **Urethrale dilatatorset met S-vormige kromming**
Gebruiksaanwijzing
- NO**
13 **Sett med S-formede urinrørsdilatatorer**
Bruksanvisning
- PL**
14 **Zestaw rozszerzacza cewki moczowej z krzywizną „S”**
Instrukcja użycia
- PT**
15 **Conjunto de dilatador uretral com curva em S**
Instruções de utilização
- SV**
16 **S-format uretralt dilatatorset**
Bruksanvisning

1

2

S~CURVE URETHRAL DILATOR SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

S~Curve Urethral Dilators are available in various sizes and configurations.

INTENDED USE

S~Curve Urethral Dilators are intended for dilation of the male urethra.

CONTRAINDICATIONS

None known

WARNINGS

None known

PRECAUTIONS

This product is intended for use by a physician trained and experienced in urethral dilation techniques. Standard techniques should be employed.

INSTRUCTIONS FOR USE

NOTE: Prior to use, immerse the AQ® dilator in sterile water or isotonic saline to allow the hydrophilic surface to absorb liquid and become lubricious. This will ease placement under standard conditions.

1. Under direct vision or fluoroscopic control, position a 0.038" wire guide through the urethra and into the bladder.
2. Introduce the smallest appropriately sized dilator, using one of the two methods below:
 - a. Hold the penis in an upward position. (**Fig 1**)
 - b. Orient the dilator so the French Size stamp on the proximal end faces **upward**, and introduce the dilator over the previously placed wire guide.

OR

- a. Hold the penis in a downward position. (**Fig 2**)
 - b. Orient the dilator so the French Size stamp on the proximal end faces **downward**, and introduce the dilator over the previously placed wire guide.
 - c. Advance the dilator to the level of the bulbar urethra.
 - d. With the dilator positioned at the level of the bulbar urethra, rotate the dilator 180 degrees so the dilator is now oriented with the French Size stamp facing **upward**.
3. Continue to advance the dilator into the bladder. Urine drainage will confirm proper placement.
 4. While maintaining the wire guide position, remove the dilator currently in place and replace it with the next appropriately sized dilator. Repeat steps 2-3 as needed, progressing from smallest to largest appropriately sized dilator until desired urethral dilation is achieved.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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© COOK 2019
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd