EN  
3

**Pigtail Ureteral Catheter Sof-Flex® AQ®**

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**Uretärlig pigtailkateter Sof-Flex® AQ®**

Bruksanvisning

## PIGTAIL URETERAL CATHETER SOF-FLEX® AQ®

**CAUTION:** U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner). Please read all instructions before using device.

### DEVICE DESCRIPTION

- Radiopaque polyurethane pigtail catheter

**NOTE:** No wire guide is provided with this set. The following is recommended: Catheters accept 0.038" PTFE wire guide.

**NOTE:** Prior to use, immerse AQ® catheter in sterile water or isotonic saline to allow the hydrophilic surface to absorb water and become lubricious. This will ease placement under standard conditions.

### INTENDED USE

The Pigtail Ureteral Catheter Sof-Flex® AQ® is intended for drainage or irrigation in the urinary tract as well as access, advancement, or exchange of wire guides.

### CONTRAINDICATIONS

These devices are contraindicated for use in patients unsuitable for catheterization.

### WARNINGS

Sterilized by Ethylene Oxide gas. Do not use if sterile barrier is damaged. This device is intended for one-time use.

### PRECAUTIONS

- This device is intended for use by physicians trained and experienced in urological techniques. Standard techniques should be employed.
- Avoid bending or kinking the catheter prior to placement. Doing so could damage the integrity of the catheter and result in patient injury.
- If you encounter resistance while advancing or withdrawing the catheter, stop. Determine the cause of the resistance before proceeding.
- Do not withdraw the catheter while it is deflected in the cystoscope/endoscope.

### POTENTIAL ADVERSE EVENTS

The adverse events which may result from ureteral catheter placement include, but are not limited to:

- Extravasation
- Hemorrhage
- Sepsis
- Edema
- Loss of renal function
- Perforation or laceration of the kidney, renal pelvis, ureter, and/or bladder
- Peritonitis
- Urinary tract infection
- Abrasion

### INSTRUCTIONS FOR USE

1. Prior to using this device, use aseptic technique to gain access to the urinary tract, leaving a wire guide in the renal pelvis. **CAUTION: Care must be taken not to pass the wire guide too far into the renal pelvis.**
2. While holding the wire guide in a set position, pass the ureteral catheter, pigtail first, over the wire guide through the ureter and into the renal pelvis.
3. Verify that the pigtail is in the correct position by standard radiographic procedure or fluoroscopy. The pigtail should be in the center of the renal pelvis.

4. Remove the wire guide while maintaining the catheter position.
5. The catheter can be used for irrigation or connected to a drainage system.

## HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

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Bloomington, IN 47404 U.S.A.