COOK

MEDICAL

CE
0088

EN 3	Shetty Gastrojejunostomy Set Instructions For Use
DA 5	Shetty gastrojejunostomisæt Brugsanvisning
DE 7	Gastrojejunostomie-Set nach Shetty Gebrauchsanweisung
EL 10	ΣΕΤ γαστρονηστιδοστομίας Shetty Οδηγίες χρήσης
ES 13	Equipo de gastroyeyunostomía Shetty Instrucciones de uso
FR 16	Set de Shetty pour gastrojéjunostomie Mode d'emploi
IT 18	Set per gastrodigiunostomia Shetty Istruzioni per l'uso
NL 21	Shetty gastrojejunostomie-set Gebruiksaanwijzing
PT 24	Conjunto de gastrojejunostomia Shetty Instruções de utilização
SV 26	Shetty gastrojejunostomiset Bruksanvisning

T _ S H E T T Y _ R E V 5

SHETTY GASTROJEJUNOSTOMY SET

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The Shetty Gastrojejunostomy Set consists of an Ultrathane® gastrojejunostomy catheter, Peel-Away® introducer, and Cope Gastrointestinal Suture Anchor Set. The set also contains an appropriately sized wire guide, dilators and other accessories for use in percutaneous placement of a gastrojejunostomy tube.

INTENDED USE

The Shetty Gastrojejunostomy Set is intended for percutaneous placement of a catheter for gastroenteric feeding.

CONTRAINDICATIONS

None known

WARNINGS

This product contains **NATURAL RUBBER LATEX**, which may cause allergic reactions.

PRECAUTIONS

- This product is intended for use by physicians trained and experienced in gastrojejunostomy techniques. Standard techniques for percutaneous placement of gastrojejunostomy tubes should be employed.
- The Shetty Gastrojejunostomy Set includes a Silhouette™ Transitionless Peel-Away introducer, which has a uniquely fabricated transition between dilator and sheath. Do not turn or pull back dilator from sheath prior to introduction. Manipulation of dilator will result in misalignment of sheath and dilator. Do not attempt to flush sheath prior to insertion.
- Withdrawal or manipulation of the distal spring coil portion of the wire guide through the needle tip may result in breakage.
- A TFE-coated wire guide must be used with this catheter.
- The potential effects of phthalates on pregnant/nursing women or children have not been fully characterized and there may be concern for reproductive and developmental effects.

INSTRUCTIONS FOR USE

Placement of Wire Guide

1. Following the Instructions for Use provided with the Cope Gastrointestinal Suture Anchor Sets, anchor the anterior wall of the stomach to the abdominal wall.
2. Using the supplied introducer needle, make a puncture adjacent to the site used to introduce the suture anchors. **NOTE:** This puncture should be performed at an approximately 45-degree angle cephalad. Use fluoroscopy to verify placement of needle within the gastric lumen.
3. Insert the supplied wire guide through the introducer needle. Under fluoroscopic guidance, manipulate the wire guide into the antrum of the stomach. Most commonly, the wire guide coils in the fundus and then exits to the antrum.
4. Remove the introducer needle and advance the seeking catheter over the wire guide.

5. Advance the wire guide and seeking catheter until the catheter tip is at the pylorus, then negotiate through the pylorus and advance the wire guide into the duodenum. **NOTE:** Take care not to coil too much seeking catheter in the fundus of the stomach. Counter forces, which make it difficult to advance the seeking catheter, may develop. Frequently, a rotational motion on the seeking catheter allows easier passage over the wire guide.
6. Continue advancing the wire guide and seeking catheter beyond the ligament of Treitz, then remove the seeking catheter.

Dilation and Placement of Peel-Away® Introducer

1. Dilate the tract through the abdominal wall using the dilators in sequential order.
NOTE: Because the stomach wall is relatively pliable, it is often difficult to be certain that the dilator is within the stomach. Ensure that each dilator enters the stomach by visualizing, fluoroscopically, the dilator curving over the wire guide positioned in the fundus.
2. Remove the dilator, leaving the wire guide in place.
3. Advance the Peel-Away introducer over the wire guide.
4. Ensure that the entire introducer is within the stomach by visualizing, fluoroscopically, the tip of the introducer beginning to curve in the fundus of the stomach.
5. Turn dilator one-half turn to break seal between sheath and dilator.
6. Pull back dilator to remove, leaving sheath portion of introducer and wire guide in place.

Placement of Gastrojejunostomy Catheter

1. Advance the gastrojejunostomy catheter over the wire and through the sheath. **NOTE:** Rotating the catheter with a pulsing motion will help facilitate passage of the catheter through the pylorus and into the jejunum.
2. To position the locking loop portion of the catheter totally within the pylorus, it may be necessary to peel away the external portion of the sheath. **It is essential that the locking loop portion be entirely within the pylorus prior to removal of the Peel-Away sheath.**
3. After catheter is fully inserted, peel away the remainder of the sheath.
4. Remove the wire guide.
5. Pull suture end tight to form loop configuration in the pylorus and inject contrast to verify catheter tip position within the jejunum.
6. Wrap suture around catheter and secure with a knot, then trim off excess suture and slide sleeve over suture to prevent leakage.
7. Attach catheter to skin surface using physician preferred method of attachment.

Unlocking Catheter Loop

1. Advance wire guide into catheter.
2. Uncover suture by folding back the latex sleeve, and release suture.
3. Withdraw catheter over wire guide.

NOTE: For catheter exchange, insert new catheter over wire guide. For removal, withdraw wire guide following removal of catheter.

Post-Placement Care

- It is not uncommon for a small amount of intraperitoneal air to be present post-procedure.
- Feedings may be instituted immediately, although it is recommended to begin with dilute feedings and advance gradually. **NOTE:** If signs of

peritonitis develop, feedings should be discontinued and appropriate antibiotics instituted. Feedings should not be resumed until all parameters are normal.

- The suture anchor(s) may be left in place for up to two weeks while tract formation occurs. Cutting the suture end(s) at the skin level releases the anchor(s) into the organ, allowing passage via the gastrointestinal system.
- If using a Luer tip syringe for flushing purposes, remove feeding adapter and connect Luer converter to the hub of the catheter. Remove Luer converter and connect the feeding adapter after flushing the catheter.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open pouches. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

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Bloomington, IN 47404 U.S.A.