Silicone Malecot Catheter

Silikonový katetr Malecot

Malecot silikonekateter

Silikon Malecot-Katheter

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T _ S M C _ R E V 1

SILICONE MALECOT CATHETER

Used to provide drainage following open renal or bladder surgeries. Malecot wings are employed to provide enhanced drainage and promote catheter retention. All-silicone construction provides maximum softness for enhanced patient comfort and provides a useful alternative to latex catheters. Intended for one-time use.

DESCRIPTION

Silicone Malecot drainage catheter

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CONTRAINDICATIONS

There are no known contraindications.

SUGGESTED INSTRUCTIONS FOR USING SILICONE MALECOT CATHETER

1. Following the open procedure, place the Malecot portion of the catheter in the area to be drained. **NOTE: Be sure the wings of the Malecot are not obstructed to ensure proper drainage.**
2. Externalize the catheter and close the surgical site.
3. Secure the catheter to the skin using suture or tape. A dressing may be applied.
4. Connect the funnel drainage lumen to a drainage bag.

CAUTION: The indwelling time should not exceed four (4) weeks. Evaluation and replacement beyond 4 weeks must be considered.

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© COOK 2019
Bloomington, IN 47404 U.S.A.