- EN**  
3 **Cook® SP Tube Introducer Set**  
Instructions for Use
- CS**  
4 **Zaváděcí souprava Cook® pro suprapubický drén**  
Návod k použití
- DA**  
5 **Indføringsæt til Cook® suprapubisk slange**  
Brugsanvisning
- DE**  
6 **Cook® SP-Katheter-Einführset**  
Gebrauchsanweisung
- EL**  
8 **Σετ εισαγωγήα σωλήνα Cook® SP**  
Οδηγίες χρήσης
- ES**  
9 **Equipo introductor de tubos suprapúbicos Cook®**  
Instrucciones de uso
- FR**  
11 **Set d'introduction Cook® pour tube sus-pubien**  
Mode d'emploi
- HU**  
12 **Cook® szuprapubikus csőbevezető készlet**  
Használati utasítás
- IT**  
13 **Set con introduttore sovrapubico Cook®**  
Istruzioni per l'uso
- NL**  
15 **Cook® suprapubische slangintroducietset**  
Gebruiksaanwijzing
- NO**  
16 **Cook® suprapubisk slange-innføringssett**  
Bruksanvisning
- PL**  
17 **Zestaw introduktora nadłonowego Cook®**  
Instrukcja użycia
- PT**  
19 **Conjunto introdutor de tubo suprapúbico Cook®**  
Instruções de utilização
- SV**  
20 **Cook® SP sondinförarsset**  
Bruksanvisning

## COOK® SP TUBE INTRODUCER SET

**CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner). Please read all instructions before using this device.**

### DEVICE DESCRIPTION

Set Contains:

- Radiopaque polyvinylchloride dilator with radiopaque TFE Peel-Away® sheath
- 2 radiopaque polyethylene dilators (8 French and 10 French)
- 18 gauge, 10 cm introducing needle
- .035, 60 cm stainless steel wire guide

### INTENDED USE

The Cook® SP Tube Introducer Set is intended for introducing a drainage catheter suprapubically into the bladder.

### CONTRAINDICATIONS

None known

### WARNINGS

Sterilized by Ethylene Oxide gas. Do not use if sterile barrier is damaged. The device is intended for one-time use.

### PRECAUTIONS

- This device is intended for use by physicians trained and experienced in suprapubic access and placement of drainage catheters. Standard techniques should be employed.
- Do not forcefully advance any component during removal or replacement of the catheter. Carefully remove the components if any resistance is encountered.
- Ensure patient has a full and palpable bladder for correct bladder position.

### POTENTIAL ADVERSE EVENTS

Complications that might occur during this procedure could include incorrect location at insertion, resulting in trauma to the surrounding tissue.

### INSTRUCTIONS FOR USE

1. Prepare puncture site in the usual manner. Advance the introducer needle suprapubically into the bladder. **NOTE:** For patients with a small suprapubic tube in place, a second puncture is not necessary.
2. Advance the wire guide through the introducer needle (or previous tract) into the bladder.
3. Once the wire guide is in place, remove the introducer needle.
4. Using a #11 blade, make a skin puncture at the midline of the wire guide exit site. **CAUTION: The blade must face caudally. (In a thin patient, the rectus sheath can be punctured at this time.)**
5. Advance dilators over the wire guide as necessary to allow the 20.0 French dilator to be introduced to the first graduation. This dilator should be introduced with the side groove facing caudally at the midline.
6. Appropriately manage pain with local anesthesia.
7. Advance a #11 blade along the side groove to open the deep site.
8. Advance the dilator through the rectus sheath and into the bladder. When dilation is complete, advance the Peel-Away® sheath into the bladder and remove the dilator.

**IMPORTANT:** Do not remove the wire guide until proper placement of the sheath is confirmed.

- Urine flow will confirm that the sheath is positioned in the bladder. Remove the wire guide. Compress the sheath between the thumb and forefinger to stop urine flow.
- Introduce a regular Foley catheter (up to 18.0 French in size) through the sheath into the bladder. Secure the catheter in the usual manner and peel away the introducer sheath.

### HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

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Bloomington, IN 47404 U.S.A.