COOK

MEDICAL

CE
0123

- EN**
3 Tunner Pediatric Urethral Dilator Set
Instructions for Use
- BG**
4 Комплект педиатрични уретрални
дилататори Tunner
Инструкции за употреба
- CS**
6 Souprava pediatrických uretrálních
dilatátorů Tunner
Návod k použití
- DA**
7 Tunner pædiatrisk dilatatorsæt til urethra
Brugsanvisning
- DE**
9 Tunner pädiatrisches Urethra-Dilatator-Set
Gebrauchsanweisung
- EL**
11 ΣΕΤ παιδιατρικών διαστολέων ουρήθρας Tunner
Οδηγίες χρήσης
- ES**
12 Equipo de dilatadores uretrales
pediátricos Tunner
Instrucciones de uso
- ET**
14 Tunneri pediaatriline ureetra dilataatori komplekt
Kasutusjuhend
- FI**
15 PEDIATRINEN VIRTSAPUTKEN TUNNER-LAAJENNINSETTI
Käyttöohjeet
- FR**
17 Set de dilateurs urétraux pédiatriques Tunner
Mode d'emploi
- HR**
19 Komplet pedijatrijskih dilatatora uretre Tunner
Upute za uporabu
- HU**
20 Tunner gyermekgyógyászati urethrális
dilatátorkészlet
Használati utasítás
- IT**
22 Set di dilatatori uretrali pediatrici Tunner
Istruzioni per l'uso
- LT**
23 Tunner pediatriinių šlaplės plėtiklių rinkinys
Naudojimo instrukcija
- LV**
25 Tunner pediatrikais urīnizvadkanāla
dilatatoru komplekts
Lietošanas instrukcija
- NL**
26 Tunner pediatriche urethrale dilatatorset
Gebruiksaanwijzing
- NO**
28 Tunner pediatric urinrørdilatatorsett
Bruksanvisning
- PL**
29 Zestaw pediatrycznego rozszerzacza Tunner do
cewki moczowej
Instrukcja użycia
- PT**
31 Conjunto de dilatador uretral pediátrico Tunner
Instruções de utilização
- RO**
33 Set dilatator uretral pediatric Tunner
Instrucțiuni de utilizare
- SK**
34 Súprava pediatrického uretrálneho
dilatátora Tunner
Návod na použitie

T _ T P U D S - M _ R E V O

SL
36

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Kullanma Talimatı

TUNNER PEDIATRIC URETHRAL DILATOR SET

Read all instructions carefully. Failure to properly follow the information provided may lead to the device not performing as intended or injury to the patient.

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner). Please read all instructions before using this device.

DEVICE DESCRIPTION

The Tunner Pediatric Urethral Dilator Set is comprised of 4 urethral dilators. The dilators are 8, 10, 12 and 14 Fr with a working length of 14 cm. The dilators have a tapered, closed end, angled tip. The French size is indicated on the end of the device without the angled tip.

Performance Characteristics

- Angled tip aids in traversing a urethral stricture.
- Tapered tip facilitates insertion into anatomy.
- Radiopaque material to aid in positioning under imaging techniques.

Device Compatibility

This device does not have a specific intended device compatibility.

Patient Population

The target patient population for these devices are pediatric patients who require dilation of the urethra.

Intended User

These devices are intended for physicians or healthcare practitioners with proper authorization to perform medical procedures (e.g., licensing, qualifications, credentials) per their local administrative and regulatory guidelines.

Contact With Body Tissue

These devices interact with the urethral tract including the urethra and bladder.

Operating Principle

These devices are introduced directly into the urethra. The dilators are introduced sequentially from smallest to largest to gradually dilate the urethra.

INTENDED USE

These devices are angled tip dilators intended for traversing and dilating urethral strictures in a pediatric patient by the sequential use of dilators of increasing French sizes.

INDICATIONS FOR USE

These devices are used for dilating and traversing urethral strictures in pediatric patients.

CLINICAL BENEFITS

Urethral dilation to widen the diameter of the urethra.

CONTRAINDICATIONS

None

WARNINGS

- The single use device is not designed for re-use. Attempts to reprocess (re-sterilize) and/or to re-use may lead to chemical contamination, device failure, and/or transmission of disease.
- Do not use the device if the sterile packaging is damaged or unintentionally opened before use
- Imaging and/or placement procedures may expose the user and patient to radiation. The user should consider potential risks associated with radiation and appropriately plan procedures to minimize exposure in patients and medical personnel. The occurrence and/or severity of undesirable side effects may be dose dependent or probabilistic.

PRECAUTIONS

No Known Precautions

POTENTIAL ADVERSE EVENTS

- Bleeding
- False passage
- Fistula
- Hematoma
- Hematuria
- Infection
- Pain
- Perforation, urethral

- Rectal injury
- Urethral injury
- Lower urinary tract symptoms (poor/diminished urinary stream, dysuria, incomplete emptying of the bladder, (recurrent) urinary tract infections, overflow incontinence, frequency, hesitancy, urgency, urinary retention, increasing post void residual, and/or pain)

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Keep dry and away from sunlight. Upon removal from package, inspect the product to ensure no damage has occurred.

INSPECTION OF DEVICE

Visually inspect the device thoroughly including all levels of the packaging (as applicable) to verify that there is no damage prior to use. Visually inspect and confirm that the integrity of the sterile barrier has not been compromised in any way.

INSTRUCTIONS FOR USE

1. Introduce the angled tip of the dilator into the urethral meatus and gently advance it beyond the obstruction.
2. Remove the dilator and repeat step 1, progressing from the smallest to the largest appropriate size.

DISPOSAL OF DEVICES

After the procedure, this device may be contaminated with potentially infectious substances of human origin and should be disposed of in accordance with institutional guidelines.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

PATIENT COUNSELING INFORMATION

Please inform the patient as necessary of the relevant warnings, precautions, contraindications, measures to be taken, and limitations of use that the patient should be aware of.

SERIOUS INCIDENT REPORTING

If any serious incident has occurred in relation to the device this should be reported to Cook Medical. This shall also be reported to the competent authority the device was used in.

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© COOK 2024
Cook Medical Europe Ltd.
William A. Cook Australia Pty. Ltd.