EN 3	<b>Urethral Dilation Balloon Catheter</b> Instruction for Use
CS 4	<b>Uretrální balónkový dilatační katetr</b> Návod k použití
DA 5	<b>Dilatationsballonkateter til urethra</b> Brugsanvisning
DE 7	<b>Ballonkatheter zur Urethradilatation</b> Gebrauchsanweisung
EL 8	<b>Καθετήρας ουρηθρικής διαστολής με μπαλόνι</b> Οδηγίες χρήσης
ES 10	<b>Catéter balón de dilatación uretral</b> Instrucciones de uso
FR 11	<b>Cathéter à ballonnet pour dilatation urétrale</b> Mode d'emploi
HU 12	<b>Húgycsőtágító ballonkatéter</b> Használati utasítás
IT 14	<b>Catetere a palloncino per la dilatazione uretrale</b> Istruzioni per l'uso
NL 15	<b>Urethrale ballondilatatiekatheter</b> Gebruiksaanwijzing
NO 17	<b>Ballongkateter for dilatering av uretra</b> Bruksanvisning
PL 18	<b>Cewnik balonowy do poszerzania cewki moczowej</b> Instrukcja użycia
PT 19	<b>Cateter de balão para dilatação uretral</b> Instruções de utilização
SV 21	<b>Ballongkateter för uretral dilatation</b> Bruksanvisning

## URETHRAL DILATION BALLOON CATHETER

**CAUTION:** U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

### DEVICE DESCRIPTION

- Set Contains (*set components may vary*)
- Urethral dilation balloon catheter
- .038 inch (0.97 mm) diameter PTFE coated stainless steel wire guide 80 cm long with 3 cm flexible tip
- Syringe

### INTENDED USE

Used to dilate the male or female urethra to treat stricture disease.

### CONTRAINDICATIONS

Do not use this device if there is bleeding or severe infection in the urethra.

### WARNINGS

- Always inflate with a sterile liquid. Never inflate with air, carbon dioxide or any other gas.
- Aspirate the balloon completely before gently removing the device. Using excessive force to withdraw the balloon can inflict trauma to tissue.

### PRECAUTIONS

- Do not overinflate the balloon.
- Balloon catheters are intended for use by physicians trained and experienced in techniques for balloon catheter dilation.
- To ensure proper regulation of balloon pressure, use of a balloon inflation device and pressure gauge is recommended.
- This device is designed for single use only. Attempts to reprocess, resterilize, and/or reuse may lead to device failure and/or transmission of disease.

### INSTRUCTIONS FOR USING URETHRAL DILATION BALLOON CATHETER

1. Remove protective sleeve from balloon and discard.
2. Insert balloon catheter tip into the urethra and gently advance balloon segment into the urethra.  
**NOTE:** lubrication of the balloon is optional. Use of lubricant may ease placement. Use of K-Y® Jelly or another water soluble lubricant is recommended.  
**NOTE:** to ensure proper regulation of balloon pressure, use of a balloon inflation device and pressure gauge is recommended.
3. Using an inflation gauge, inflate the balloon to a pressure of 90 psi, or less.
4. Inflate for 20 to 40 seconds, or until desired dilation is achieved.
5. Deflate balloon by syringe aspiration; remove and discard.  
**NOTE:** aspirate the balloon completely before gently removing the device. Using excessive force to withdraw the balloon can inflict trauma to tissue.

### INSTRUCTIONS FOR PLACING URETHRAL DILATION BALLOON CATHETER OVER-THE-WIRE

1. Remove protective sleeve from balloon and discard.
2. Insert wire guide into bladder.

3. Load catheter onto wire guide. Gently advance catheter over wire into the urethra.

**NOTE:** lubrication of the balloon is optional. Use of lubricant may ease placement. Use of K-Y® Jelly or another water soluble lubricant is recommended.

4. Remove wire guide.

5. Using an inflation gauge, inflate the balloon to a pressure of 90 psi, or less.

6. Inflate for 20 to 40 seconds, or until desired dilation is achieved.

7. Deflate balloon by syringe aspiration; remove and discard.

## HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

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© COOK 2020
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd