Ureteral Dilator Set

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EN
3
CS
3
DA
4
DE
5
EL
5
ES
6
FR
7
HU
7
IT
8
NL
9
NO
9
PL
10
PT
11
SV
12
ZH
12

URETERAL DILATOR SET

Used for dilation of the ureter prior to ureteroscopy and/or stone manipulation. Intended for one-time use.

SET CONTAINS

- Nine radiopaque polyethylene dilators 60 cm long
- .038 inch (0.97 mm) diameter stainless steel wire guide 145 cm long with 3 cm flexible tip

NOTE: Prior to use, immerse AQ® dilators in sterile water or isotonic saline to allow the hydrophilic surface to absorb water and become lubricious. This will ease placement under standard conditions.

CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS

There are no known contraindications.

SUGGESTED INSTRUCTIONS FOR USING URETERAL DILATOR SET

1. Under x-ray or fluoroscopic control, cystoscopically pass the wire guide the desired distance into the ureter. (The safety index mark, 64 cm from the wire guide's flexible tip, will be externally visible under normal circumstances.)
2. Progressing from the smallest to the largest appropriate size, pass the radiopaque dilators over the wire guide while maintaining the wire guide's position. **NOTE:** Dilators up to approximately 12.0 French may be passed through the working channel of the cystoscope. However, the telescope and inner scope elements must be removed—leaving the sheath—to accommodate the larger sizes.

CAUTION: Do not advance dilators beyond the 64 cm safety index mark. When the end of the dilator is at the mark, a 4 cm length of wire protrudes from the tip. Further advancement of the dilator will override the wire guide.

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© COOK 2019
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd