**Urethral Dilator Set**

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**Dilatatorsæt til urethra**

**Urethra-Dilatator-Set**

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## URETHRAL DILATOR SET

Used for urethral dilation. Intended for one-time use.

### SET CONTAINS

- Radiopaque dilators 37 cm long
- .038 inch (0.97 mm) diameter stainless steel wire guide 80 cm long with 1 cm flexible tip

**NOTE:** Prior to use, immerse AQ® dilators in sterile water or isotonic saline to allow the hydrophilic surface to absorb water and become lubricious. This will ease placement under standard conditions.

**CAUTION: Sterile if the package is unopened or undamaged. Do not use if package is broken.**

**CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.**

### CONTRAINDICATIONS

There are no known contraindications.

### SUGGESTED INSTRUCTIONS FOR USING URETHRAL DILATOR SET

1. Advance the wire guide directly from the enclosed holder.
2. Introduce the flexible tip of the wire guide into the urethral meatus, and gently manipulate it beyond the obstruction and into the bladder. (The safety index mark, 39 cm from the wire guide's flexible tip, will be externally visible under normal circumstances.)

**NOTE:** The wire guide can also be placed through a cystoscope and maintained in position as the scope is removed.

3. Progressing from the smallest to the largest appropriate size, pass the radiopaque dilators over the wire guide while maintaining the wire guide's position.

**CAUTION: Do not advance dilators beyond the 39 cm safety index mark. When the end of the dilator is at the mark, a 2 cm length of wire protrudes from the tip. Further advancement of the dilator will override the wire guide.**

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© COOK 2019
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd